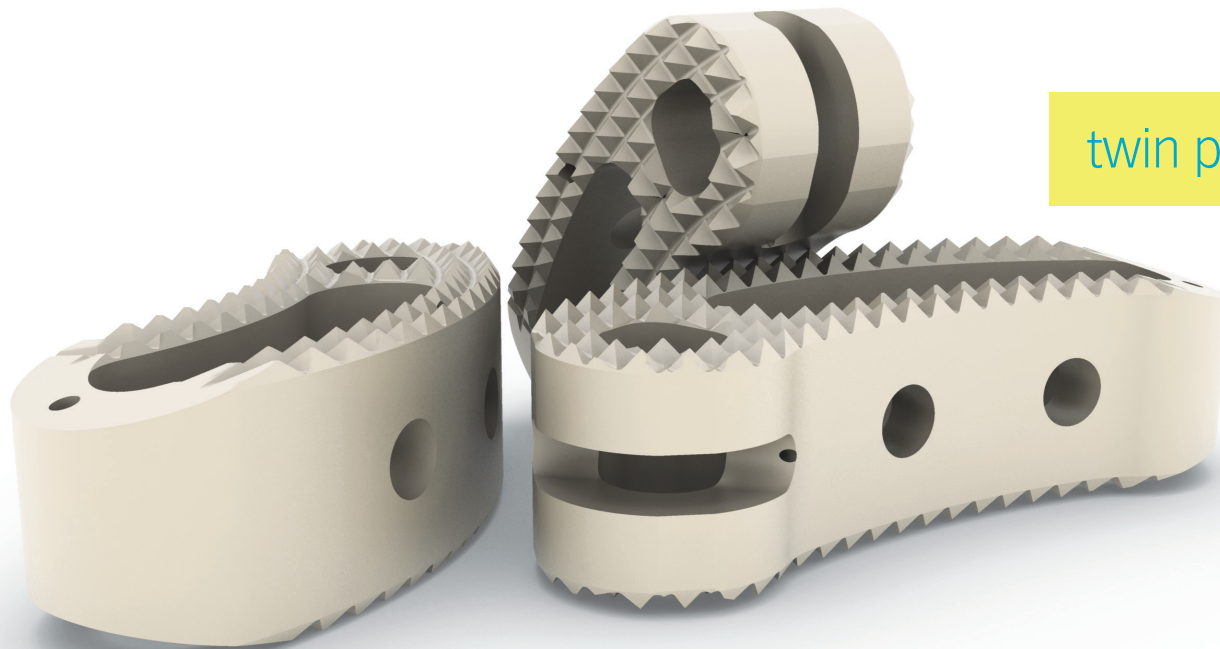
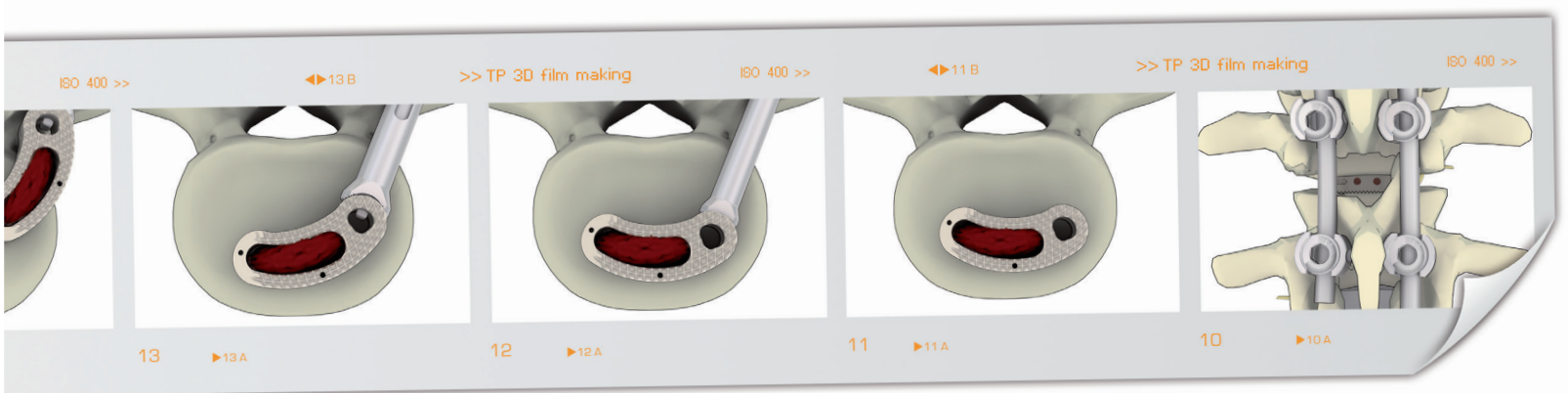
twin peaks®

TLIF

surgical technique

Introduction

TLIF cages are indicated for transforaminal lumbar interbody fusion.

Materials

Twin Peaks cages are made of PEEK-OPTIMA® by Invibio® (A radiotransparent material with an elasticity close to that of bone).

The cages have radiopaque tantalum markers.

Design rationale

Characteristics of Twin Peaks TLIF cages:

- The implant has been designed for use with the Twin Peaks TLIF surgical technique:
 - with three widths: 25 mm; 30 mm; 35 mm
 - with eight heights: from 7 mm to 13 mm
 - with two lordotic angles: 4° and 9°
 - restores the natural lordosis
- Tapered end:
 - for easy insertion.
- Radiopaque markers:
 - show the position of the cage.
- Pyramidal teeth:
 - for high primary stability
 - prevent posterior migration
- Graft chamber and vascularization openings to encourage bone regeneration.

Indications

- Degenerative discopathy, instability
- Acquired or congenital spinal deformation (scoliosis, cyphosis, lordosis)
- Failure of previous intervention
- Trauma.

Precautions for use

Implants and instruments must only be used by qualified professionals. It is essential to read and absorb the information provided in the instruction sheet enclosed with the implants.

The fitting of Twin Peaks - TLIF cages must systematically be accompanied by a posterior fixation such as the mont blanc® system.

Contra-indications: cf. Instructions for use twin peaks®.

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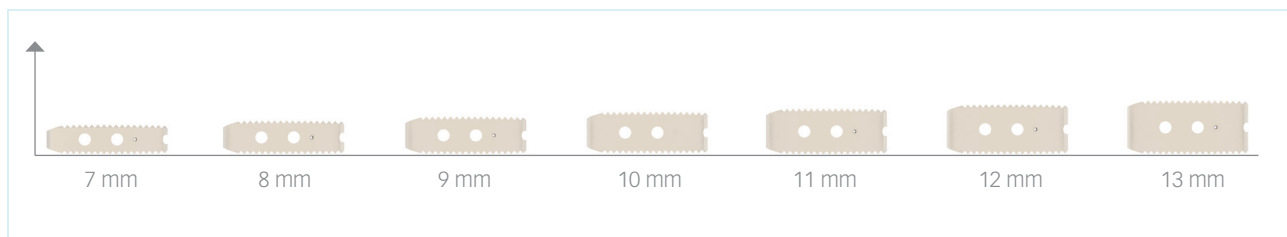
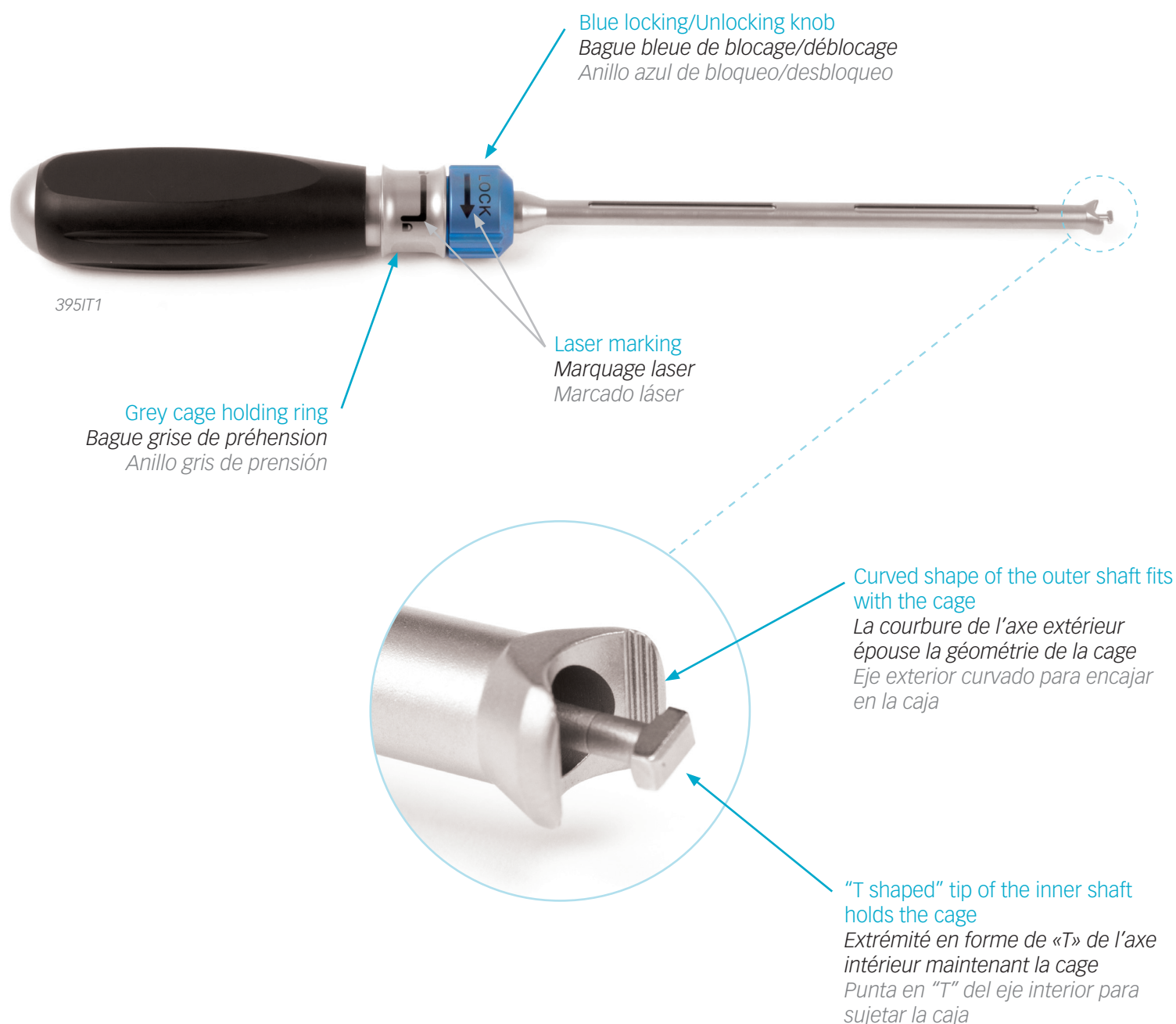
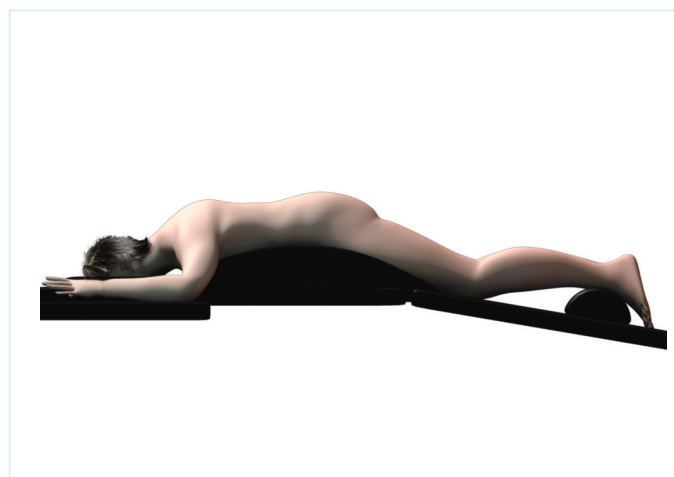
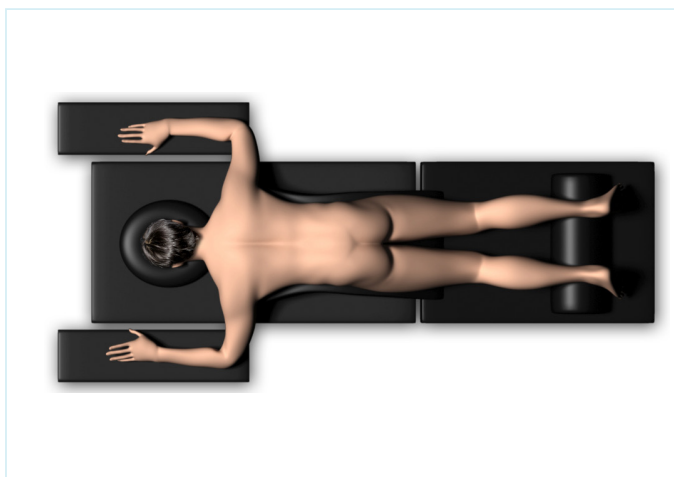
Posterior approach

Position of the patient

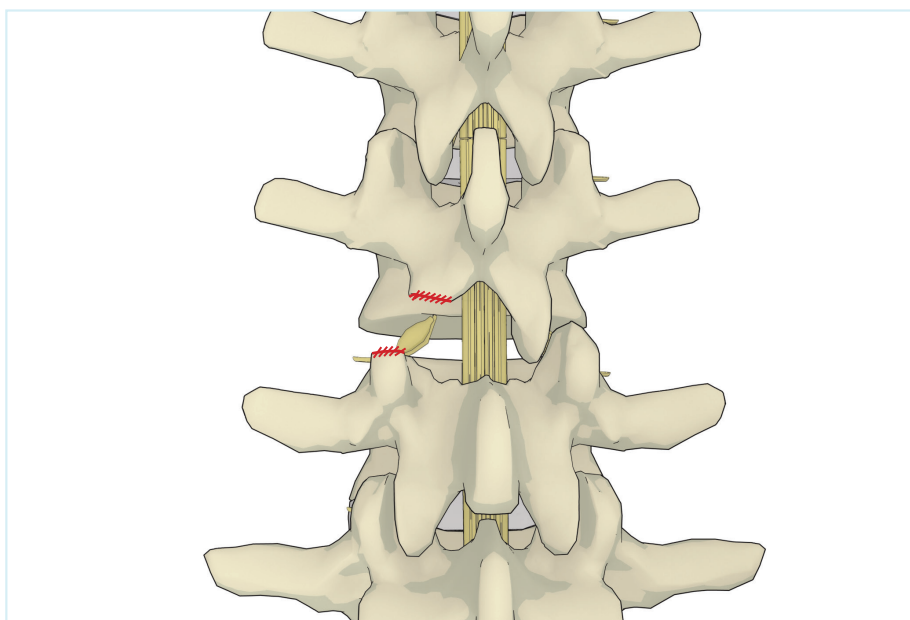
The patient is generally positioned in ventral decubitus avoiding compression points. The intervention is carried out by transforaminal route; median or paramedian (Wiltse).

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The resection of articular apophyses must be sufficient for the passage of the cage into the intervertebral space.

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Site preparation

intervertebral space distraction

Distraction using pedicle screw heads or lamina spreader

The surgeon can decide whether distraction of the intervertebral space is possible using:

- a lamina spreader: clamp P/N 394DLS1,
- or pedicle screw heads: clamp P/N 057DC1 (supplied with the *mont blanc* kit).

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* Kit Mont Blanc ref. C1MB01.

Site preparation

intervertebral space distraction

Distraction using lumbar distractors or distractor reamers

Intervertebral space distraction using distractor reamers to:

- restore the intervertebral height
- **AND** prepare the vertebral endplates.

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* Kit Twin Peaks P/N C1TP03-1
The quick connect handle is required for using standard distractors.

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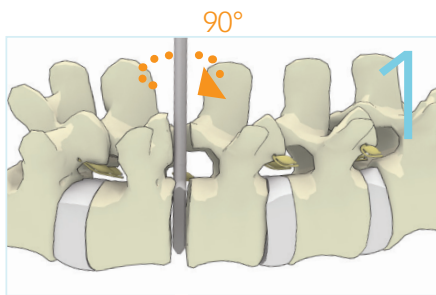
Site preparation

intervertebral space distraction

3 steps distraction

The distractors are inserted on their flat side into the disc space. Start with the smallest lumbar distractor or distractor reamer (size 7).

Step 1

Turn the distractor by a quarter-turn, i.e. 90°.

Step 2

If the space is not filled, withdraw it, then insert the next upper size.

Step 3

The optimum distraction is obtained where the intervertebral space is restored, and when a tensioning of the surrounding ligamentous apparatus is felt.

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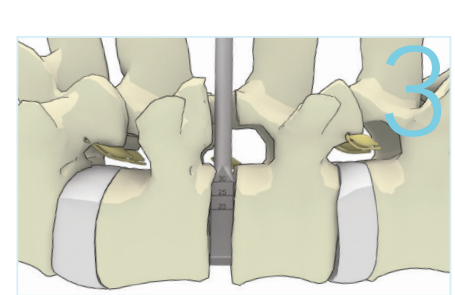
Cleaning the intervertebral space

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Preparing the endplates

Special devices

- standard curette ref 079PRC1, angled curettes P/N 396PAL1 and 379PAS1.
- and/or reamers P/N 376RL7S1 to 069RL13L1
- and/or distractor reamers P/N 387DL7R1 to 393DL13R1.

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* Kit Twin Peaks P/N C1TP03-1

The quick connect handle is required for using reamers.

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Determination of the intervertebral dimensions

Sizers

Removable handle, quick connect
Manche amovible, encliquetable
Mango amovible, conectable

380DL7T1 to 386DL13T1

Sizers L 25 / H 7 -> 13 / 4°
Gabarits L 25 / H 7 -> 13 / 4°
Pruebas L 25 / H 7 -> 13 / 4°

Sizer

The final height of the implant is determined by the largest sizer that can be inserted into the intervertebral space.

1 length = 25 mm, 1 angle = 4°, in 7 heights (7 mm to 13 mm) to match the height of the implants.

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* Kit Twin Peaks P/N C1TP03-1
The quick connect handle is required for using sizers.

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395IT1

- Ensure that the impactor is in open position: "Cage prehension" by pushing and turning the grey cage holding ring following the direction 

TIP: "T-shaped" tip largest dimension **ALIGNED** with the flat of the handle.

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Step 1

- Insert the "T-shaped" tip into the cage slot. (Picture 1)

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Step 2

- Secure the cage by turning the grey holding ring following the . Two flats on the cage are used to locate the holding position naturally (picture 2).

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Step 3

- Lock the cage rotation by turning the blue locking/unlocking knob following locking  Symbol. (Picture 3)

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Cage locked

Graft filling

The lumbar cage selected is positioned on the impactor, then placed on the support. The cage is then filled with autograft or bone substitute.

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Spineway can supply 2 ranges of synthetic bone substitute.

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Sonora: Calcium phosphate ceramic granules.

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Further information can be found in the respective brochures.

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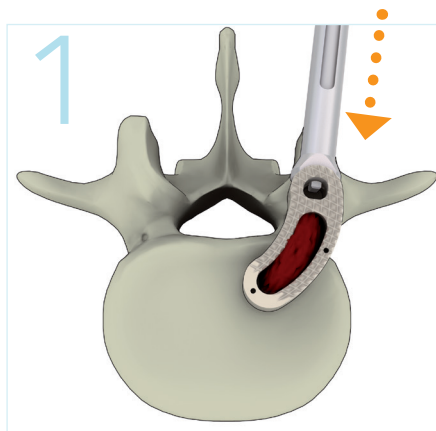
Secure and mobile

Step 1: Cage insertion

Introduce the cage into the intervertebral space in a straight direction. (Picture 1)

NB: Check that the cage cannot rotate by turning the locking/unlocking knob following **LOCK** → symbol.

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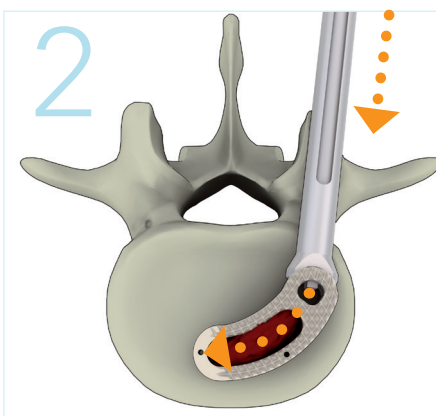
Step 2: Cage unlocking

Unlock the cage by turning the blue locking/unlocking knob backwards relative to the **LOCK** → direction.

Step 3: Cage impaction

Impact the cage freehand as the cage can pivot with respect to the impactor. (Picture 2)

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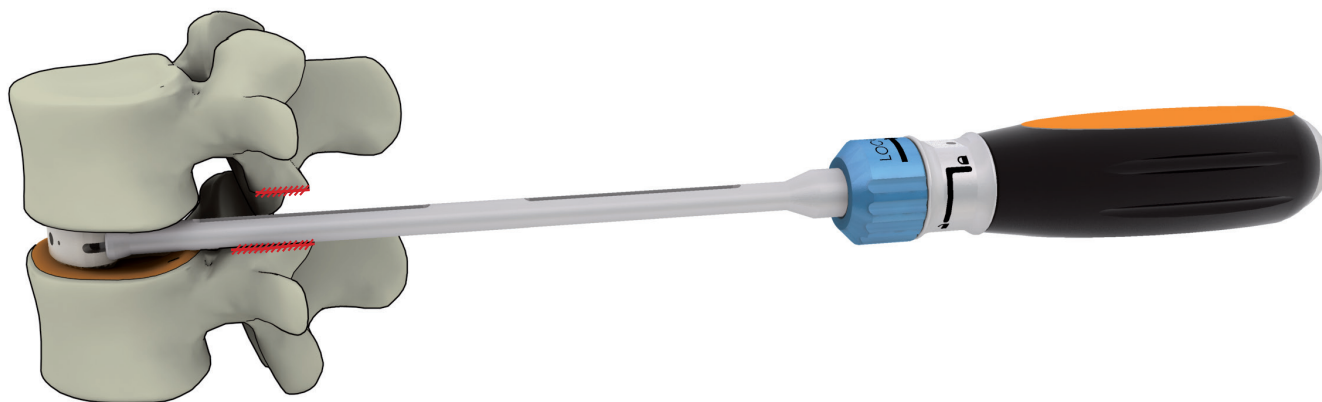
3

Step 4: Impactor removal

Release the cage by turning the grey cage holding ring following the  direction and remove the impactor. (Picture 3)

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Check that the flats on the impactor handle are parallel with the vertebral endplates and then remove the impactor. (Picture 3)

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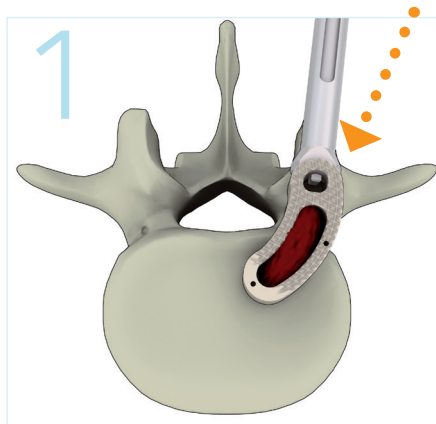
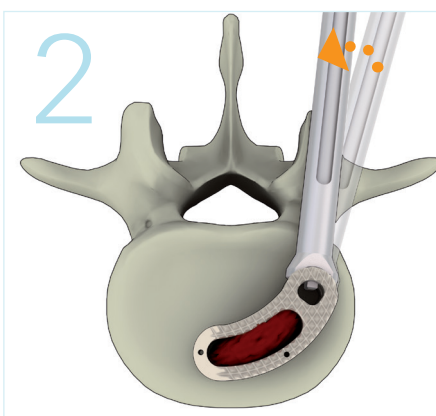
Secure & mobile

Step 2: Cage unlocking

Unlock the cage by turning the blue locking/unlocking knob backwards relative to the **LOCK** direction.

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Step 3: Cage impaction

Cage is successively:

- locked,
- impacted,
- unlocked using the blue locking/unlocking knob (symbol **LOCK**) in order to re-align the impactor in the desired angulation until a satisfactory position is achieved. (Picture 2)

ALTERNATION secured & mobile then locked

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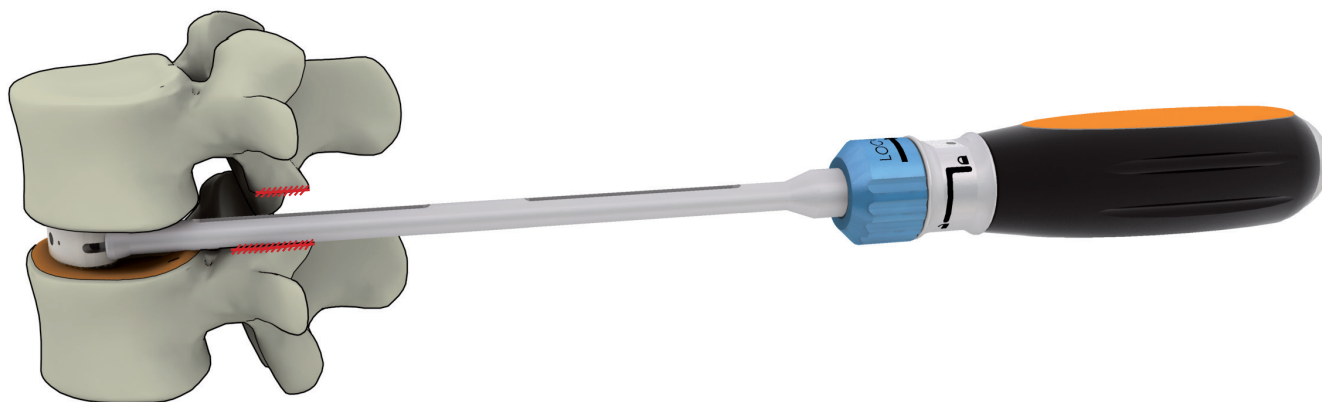
3

Step 4: Impactor removal

Release the cage by turning the blue locking/unlocking knob backwards relative to the **LOCK** direction, then by turning the grey cage holding ring following the **LOCK** direction.

⚠ Check that the flats on the impactor handle are parallel with the vertebral endplates and then remove the impactor. (Picture 3)

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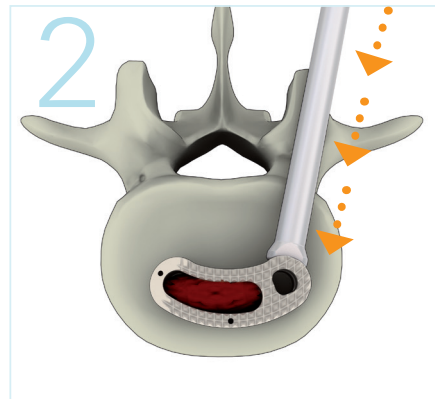
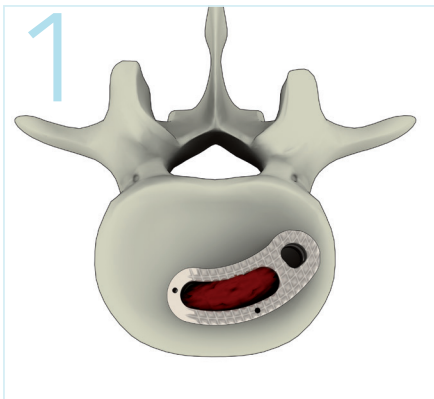
Final impactor
Impacteur de finition
Impactor de acabado

426IFT1

Final impaction readjustment of the cage

Use the final impactor to:

- Complete the positioning of the cage
- Make an antero-posterior adjustment if necessary. (Picture 2)

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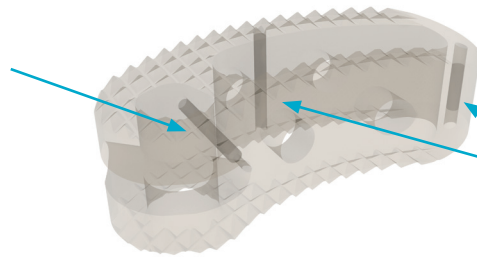
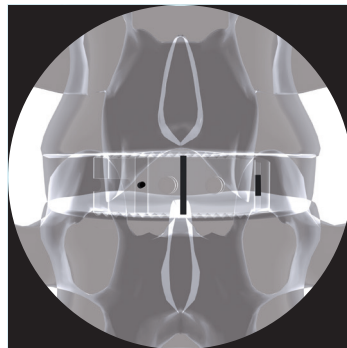
Ensure the longest marker is aligned with medial plan.

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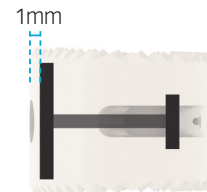
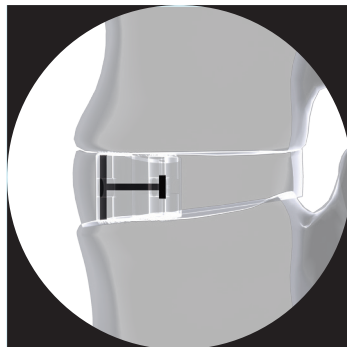
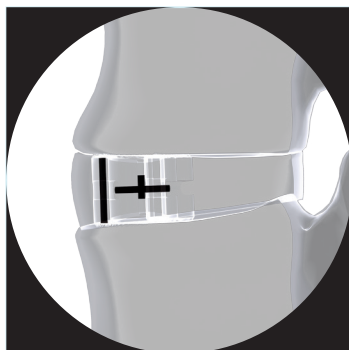
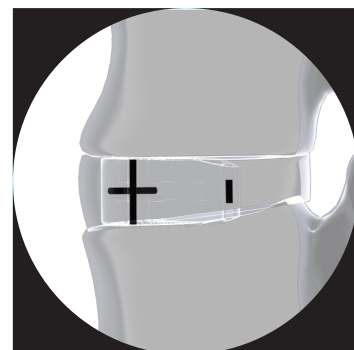
"H" Symbol:
desired position

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Símbolo "H":
Posición deseada

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UNDER ROTATED:
PUSH the cage to achieve desired position.

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OVER ROTATED:
"PULL BACK" the cage to achieve desired position.

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Posterior fixation

Posterior fixation

The implantation of cage Twin Peaks TLIF must be accompanied by a posterior fixation by pedicular anchoring.

You can use the operating technique of the **mont blanc**® system for this.

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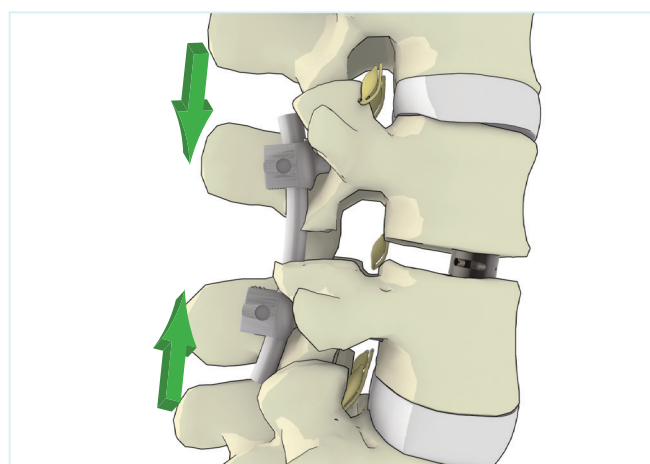
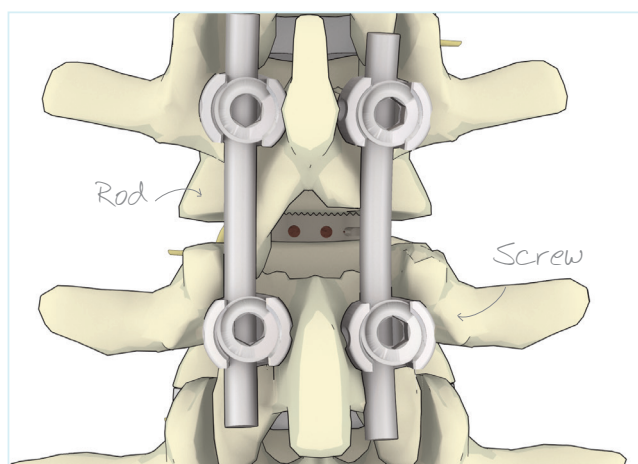
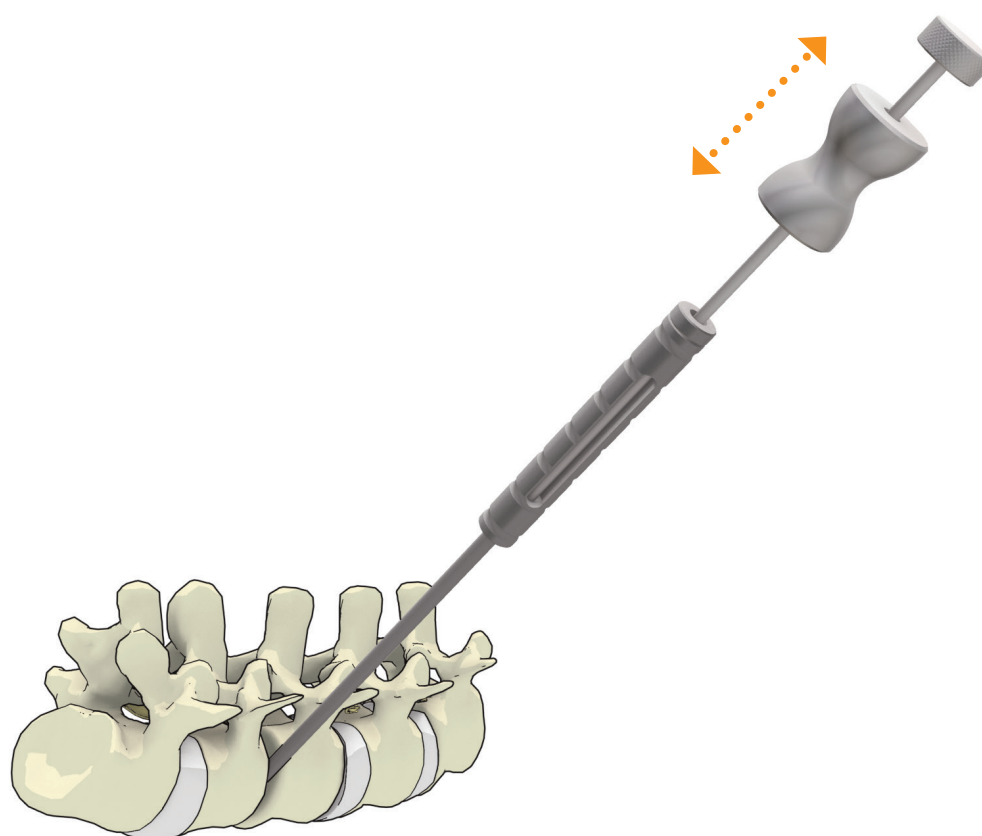
Holder
Préhenseur
Porta implante

026HLC1*

- Slide the slap hammer onto the holder, then screw the holder into the handle of the extractor (P/N 423HET1).

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- Insert the "T-shaped" tip of the extractor in the cage slot.

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- Move the slap hammer from front to back to remove the cage.

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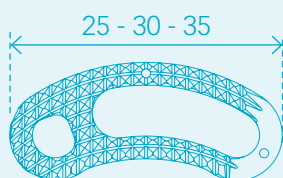
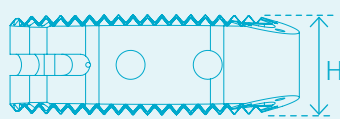
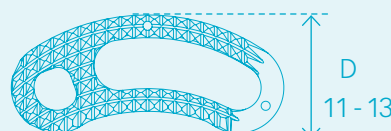
7 -> 13

Depths - Profondeurs - Profundidad

Height, length, depth are measured in millimeter (mm)

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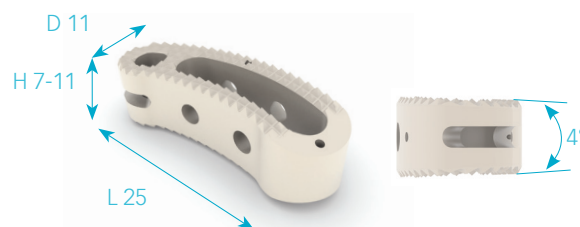
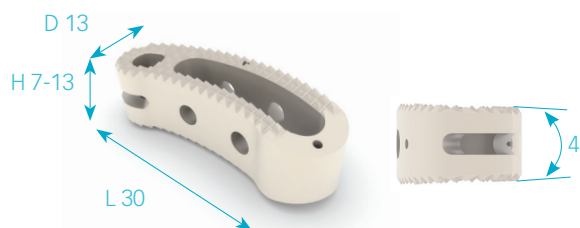
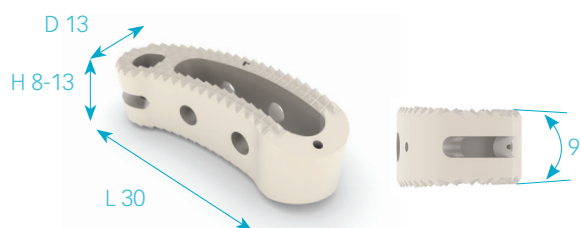
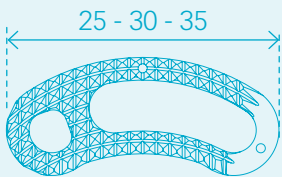
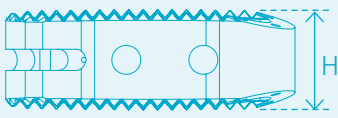
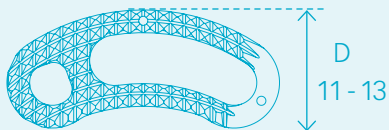
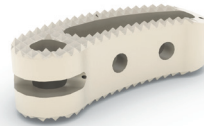
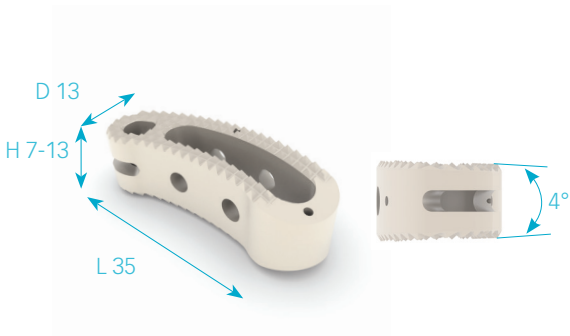
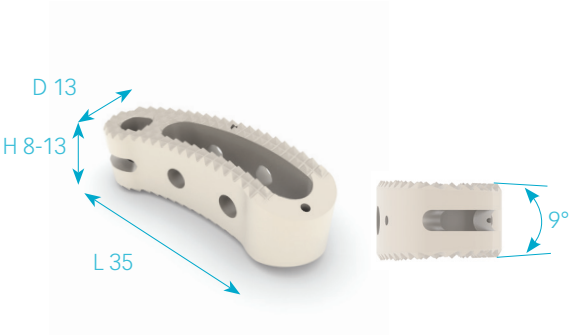
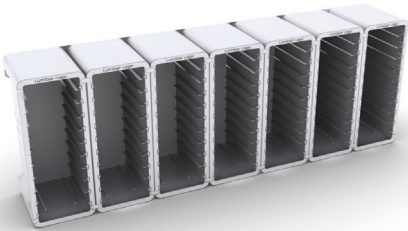
Sterile - Stérile - Estéril

Characteristics / Consequences:

- Traceability of the implant is ensured: peel-off labels.
- No need for iterative sterilisation in the autoclave.

Operating room personnel: product codes can be read directly from the box.

Implant sterilisation by gamma radiations is guaranteed for 5 years.

The implants are packed in a cardboard box (140 mm x 95 mm x 20 mm), covered with film (so the boxes can be cleaned before being taken into the operating room).

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SPINWAY, twin peaks®, mont blanc®, sonora®, neve® are registered trademarks of Spineway company.