33 DALIS
BVPŽ 39254100-8

VIBRUOJANTIS ŽADINTUVAS
mod. SBP100

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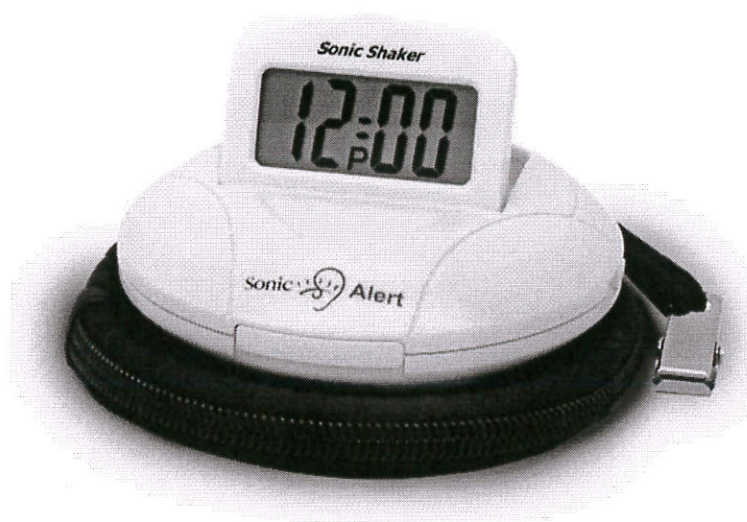
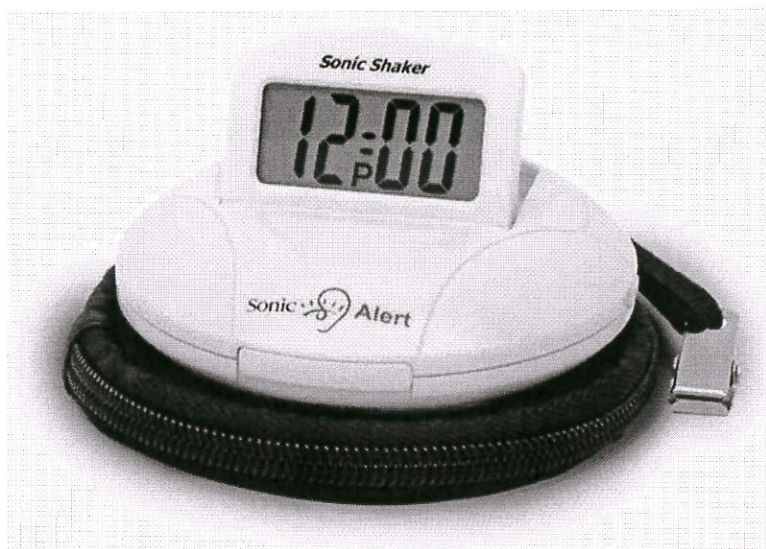
TECHNICAL SPECIFICATIONS

- Standard size batteries included (2 AA ; 1 AAA)
- Has new green power technology, so now uses 60% less energy!
- Size : 10 x 10 x 3 cm
- Weight: 160 g
- Travel case with pillow clasp included

SonicAlert

Items pictured and details are subject to change at the manufacturer's discretion.

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Sonic Alert

America's Leader in Visual Alerting Systems

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBP100 is warranted against manufacturing defects in materials and workmanship for one (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBP100 without charge for parts and labor. Simply send the SBP100 (postpaid) and a copy of your sales slip as proof of purchase to:

Sonic Alert Inc., 1050 E. Maple Road, Troy MI 48083

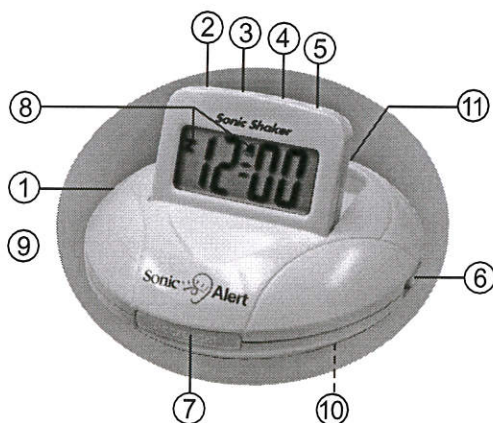
Sonic Shaker Portable Vibrating Alarm Clock Model SBP100

GENERAL DATA

Power: DC 3V (1.5Vx2 pieces AA SIZE) &
DC1.5V (1.5Vx1 pieces AA A SIZE)

PORTABLE VIBRATING ALARM CLOCK SBP100 DIAGRAMS & OPERATING INSTRUCTIONS IMPORTANT - Please read these instructions carefully before use and retain for future reference.

- | | |
|-----------------|--------------------------|
| 1. ALARM ON/OFF | 6. ALARM MODE SWITCH |
| 2. TIME SET | 7. SNOOZE & LIGHT BUTTON |
| 3. ALARM SET | 8. ALARM INDICATOR |
| 4. HOUR | 9. PM INDICATOR |
| 5. MINUTE | 10. BATTERY COMPARTMENT |
| | 11. TEST BUTTON |

INSTALLATION

(See diagram for additional directions for settings)

1) HOW TO INSTALL BATTERIES

- 1.1 Remove the BATTERY COMPARTMENT (10), insert 1 AAA battery for the clock and insert 2 AA batteries for the vibrator / alarm.
- 1.2 To insert the batteries, please observe the correct position of the polarity, as marked inside the battery compartment.

2) WHEN TO REPLACE BATTERIES

- 2.1 When the LCD screen becomes dim, it is time to change the AAA battery.
- 2.2 When alarm sound quality becomes poor or volume is low or the vibrator is vibrating very slow, it is usually an indication of weak batteries. Replace the weak batteries to avoid any possible leakage.

3) MAINTENANCE

- 3.1 If the unit is not used for long period of time, remove all batteries to avoid any possible leakage.
- 3.2 When traveling with this unit it is advised to remove the batteries for the vibrator/alarm (2 AA size) to avoid draining the batteries in the event light button is accidentally pressed during travel.

SETTING CLOCK TIME AND ALARM TIME (see diagrams)

4) SETTING CLOCK TIME

- 4.1 Press and hold the TIME SET button (2), then press the Hour button (4) until the correct hour is displayed. Remember to set the correct AM or PM. When PM time is registered, a "P"(9) will appear on the display.
- 4.2 Press and hold the TIME SET button (2), then press the Minute button (5) until the correct time is displayed.

5) SETTING ALARM TIME

- 5.1 Press and hold the ALARM SET button (3), then press the Hour button (4) until the correct hour is displayed. Remember to set the correct AM or PM. When PM time is registered, a "P"(9) will appear on the display.
- 5.2 Press and hold the ALARM SET button (3), then press the Minute button (5) until the correct time is displayed.

ALARM OPERATION (see diagrams)

6) Set the alarm time as indicated in Setting Alarm Time.

- 6.1 Set the Alarm ON/OFF Switch (1) to ON. Alarm indicator will appear in the top center of the display (8) indicating the alarm function is on. The alarm is now set to turn on automatically at the selected time.
- 6.2 You can select the alarm mode for waking, i.e. buzzer, vibrator, or buzzer and vibrator by setting the Alarm Mode Switch (6) to your desired position.
- 6.3 With the alarm sounding, press the ALARM OFF (3) to turn off the alarm. The alarm comes on again the next day at the same time.
- 6.4 With the alarm sounding, press the Snooze Button (7) after the alarm sounds, the alarm will go off and will come on again in approximately 4 minutes. You may repeat this cycle as many times as desired.
- 6.5 To stop the alarm permanently set the Alarm ON/OFF Switch (1) to OFF.

Night Light (see diagram)

- 7) To read the clock time in the dark, press and hold the Light / Snooze Button (7).

Test Button

- 8) The test button (11) is used to demonstrate the buzzer and vibrator without setting the alarm. To use the test button, slide the function switch to the desired position i.e. buzzer, vibrator or buzz/vib and press the test button to experience the buzzer or the vibrator.

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