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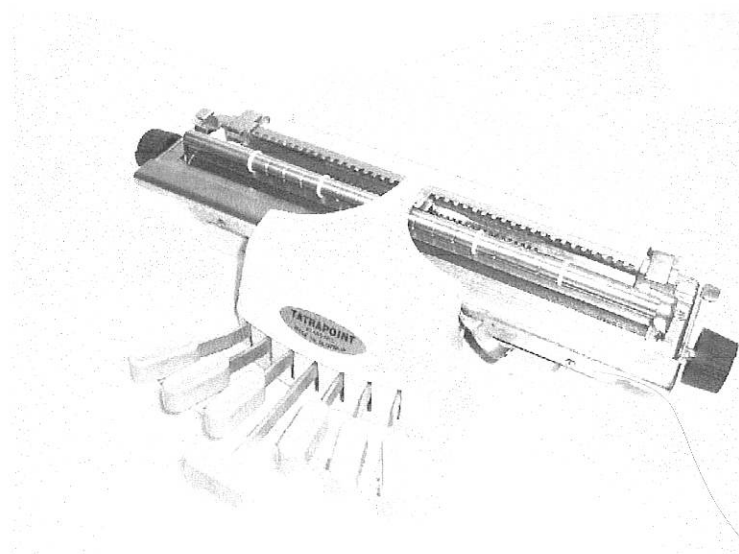
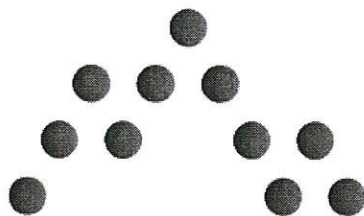
STANDARD

INSTRUCTION MANUAL

MECHANICAL BRAILLE TYPEWRITING MACHINE

FOR THE BLIND

The machine is delivered in the carrying case. In the carrying case it is fixed in a foam against movement during transportation. The carriage (4) is fixed against movement by margin stops (9, 10) placed in the centre of the carriage next to the embossing mechanisms.

Remove the machine from the carrying case and place it on a flat horizontal surface. The base is coated in rubber to prevent slipping. The margin stops are located on the upper back edge of the carriage. Push the right margin stop (10) towards the keyboard and move it to the right. Move the carriage a little bit to the left (e.g. by spacebar). Push the left margin (9) and to move it to the left. Before returning the machine into the carrying case, please fix the carriage against its movement. Use the above described operations in the reversed order.

Now set the left and the right margin stops to the positions corresponding to the width of the used paper and the required number of characters in the line. The largest usable format of paper allows writing 40 characters in one line (depending on the width of the carriage) and 25 lines per page. The bell will ring five characters before the end of a line that was set up by the margin on the right side.

To insert a paper (210 x 297 mm), turn the roller (6) from you using the turning knobs (8) until its final position. When the roller stops, it is not possible to turn it anymore (if you will turn it with power you can damage it). Right under the pressing roller (5) there is a gap in the roller (6) from the machine back side. It is a place to insert the paper. Hold paper with one hand and slide it into position, so that bottom edge of the paper is in contact with the vertical stop surface. If necessary adjust by sliding the paper gently to the left or right. Finally, while still gently pressing the paper towards you with one hand, turn the turning knob (8) towards you and the roller will close and fix the paper in its position.

Now feed the paper in the machine by turning the turning knobs towards you until the roller stops. (After it stops do not force it to turn, you can damage it.) In this position you can start to write first line. While moving the turning knobs the pressing roller (5) with rubber rings (30) is also turned. The precise movement of paper is ensured by cogwheels (26, 27) on both the paper and the pressing roller what protects paper against folding. The firm clamping of the paper, its precise movement in both directions and rubber rings on the pressing roller enables you to make corrections in the text without crushing the dots. To move the paper to the next line, move the turning knobs away from you. A "click" can be heard that indicates the roller has been turned up one braille line. Then push the carriage (e.g. by pushing the right turning knob) as far left as it will go to come to the beginning of the next line (hold the matrix (2) for support).

When you will finish the last line (25. lines / page), turn the feeding knobs from you. The roller will open and release the paper.

Raised dots of a Braille characters are embossed by simultaneous pressing of corresponding key buttons (17). Distance between Braille cells is 6 mm, distance between dots in the Braille cells is 2,7 mm. Allocation of the Braille dots and its keys is following:

The standard model (for both hands writing): The middle (long) key serves as a spacebar. From the middle key to the left there are the keys for dots 1, 2 and 3. From the middle key to the right there are the keys for dots 4, 5 and 6.

The right hand model (for one hand writing): The allocation is the same as for the standard model, but the functions of the key for dot 2 and spacebar are mutually interchanged. The length of the middle key is adjustable for this model.

The left hand model (for one hand writing): The allocation is the same as for the standard model, but the functions of the keys for dot 5 and spacebar are mutually interchanged. The length of the middle key is adjustable for this model.

There are carriage release levers (15) on the upper back ends on both sides of the carriage. When pressing the release lever hold the carriage to prevent it from an uncontrolled movement.

There is a backspace key (14) on the right side of the matrix (14) behind the key buttons. The backspace key enables to move paper one character back against the direction of writing.

On both ends of the upper surface of the carriage next to the ends of pressing roller are placed special holders for DYMO tape (22). You can fix the tape in here and write on it.

The structure of the braille creates the set of pressed and turned parts assembled in one stable unit, which ensures the trouble-free operation. The housing is closed preventing the insides from any mechanical damage and dirt. The moving parts of the braille are preserved by oil and grease. We recommend after the long-term working to clean up the visible surfaces of moving parts with alcohol and to wipe out by oil. In case that some parts of the braille damages you should contact your service centre or directly the producer.

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
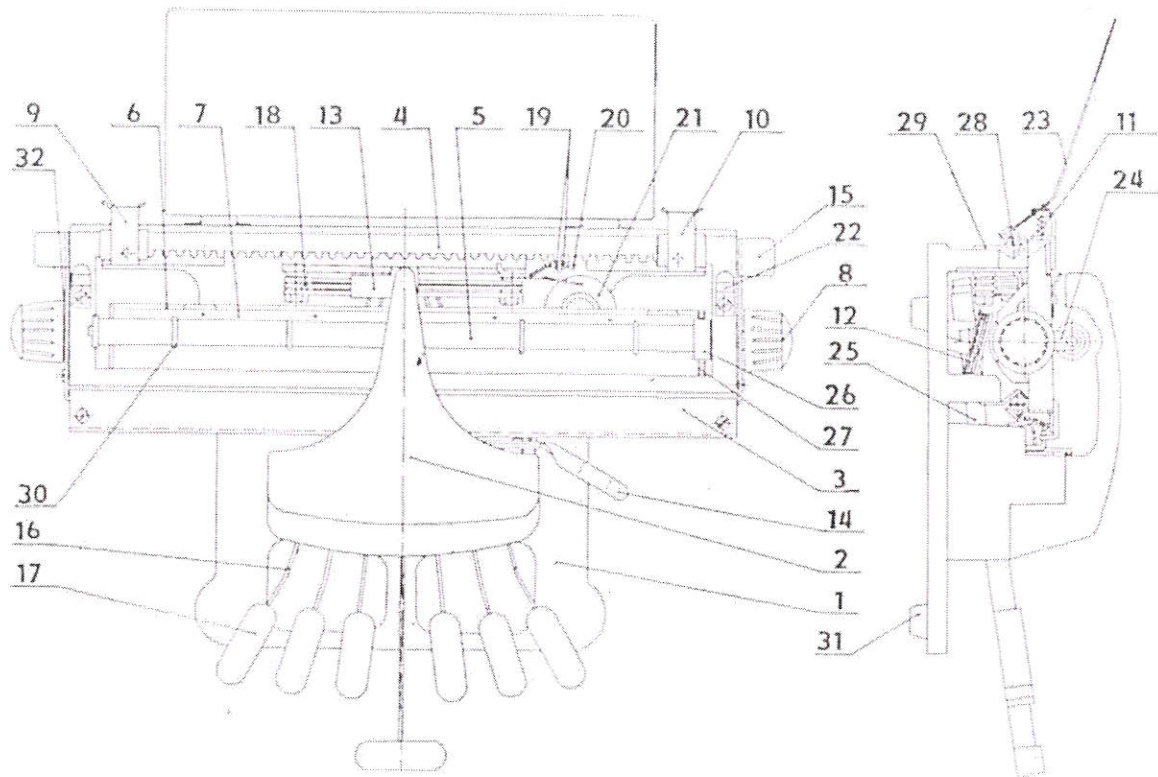
www.facebook.com/tatrapoint

TATRAPOINT STANDARD - List of parts

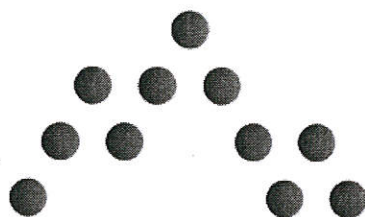- | | |
|---------------------------|------------------------------|
| 1 Base | 18 Arm |
| 2 Matrix | 19 Nut |
| 3 Comb | 20 Screw |
| 4 Carriage | 21 Bell |
| 5 Pressing roller | 22 DYMO tape holders |
| 6 Roller | 23 Reading desk |
| 8 Turning knobs | 24 Holder of pressing roller |
| 9 Margin stops | 25 Returning support |
| 10 Margin stops | 26 Cogwheel |
| 11 Spring | 27 Cogwheel |
| 12 Drum | 28 Ball bearing |
| 13 Needle holder | 29 Bar |
| 14 Backspace | 30 Rubber ring |
| 15 Carriage release lever | 31 Feet |
| 16 Keys | 32 Stud |
| 17 Key buttons | |

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