Labor Diagnostik Systeme GmbH

Infanteriestr. 19, Geb. 6

D-80797 München / Germany

Capillary Blood Collection System with Click-Clack Closure and membrane in the lid (KBS500CC, DS)

Twist and Turn Closure (Click-Clack)

Technique avoids aerosol effect and lowers risk of infections

Allows quick and clean opening by a short clockwise rotation

A numerical scale easily helps to track the blood volume (max. 500 µl.).

Inner surface of the tube and the capillary are evenly coated, i.e. immediate contact of the blood with anticoagulant EDTA during draining the blood into the capillary.

Coated capillaries of unbreakable plastics are available in 75 to 250 µl.

For all conventional testing:

blood count (EDTA)

Serum extraction

Blood sugar (NaF)


Plasma extraction (Li-Heparin)

Colour coded for easy identification of blood samples. Two different colour codes are available (national and international).

Alternative product to sarstedt Microvette and Kabevette.

Operating Instructions:

Fully fill the capillary tube. Turn collection tube upright. After draining shake remaining blood out of the capillary. Remove the capillary and discard it. Close the container with the attached stopper and immediately mix well. Tube could be used in autoreloader not open the lid in closure or manually when aspiration needle move into tube.

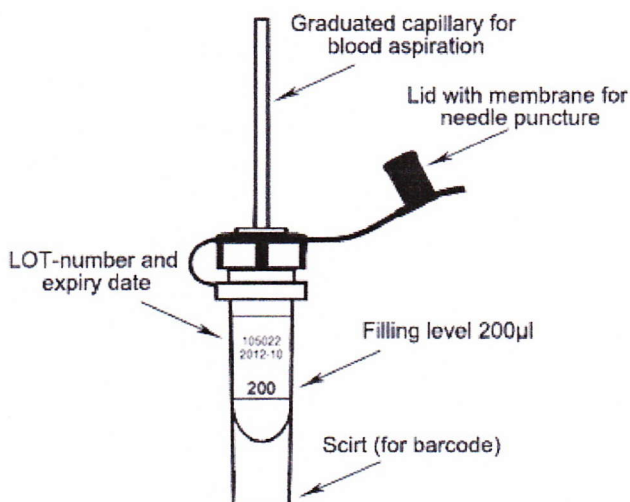This product fulfills the requirements of the European Medical Product Law and the European Directive 98/79/EC for in vitro diagnostic medical products.

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SC Sanguis Counting Kontrollblutherstellungs- und Vertriebs GmbH • Im Löchelchen 11 • D-51588 Nümbrecht • Germany
SC Sanguis Counting Kontrollblutherstellungs – und Vertriebs GmbH • Löchelchen 11 • D-51588 Nümbrecht •