

Oosafe®

IVF LABORATORY DISINFECTANTS

Manufactured in accordance  
with EN ISO 13485:2016


DISIN  
FECT  
ANTS



\*All enveloped viruses from Annex A, EN14476:2013+A1:2015 including HBV, HCV, HDV, HIV, Coronavirus, Influenza, Rabies, Measles virus.

# Oosafe® IVF LABORATORY DISINFECTANTS

## Oosafe® DISINFECTANT FOR CO<sub>2</sub> INCUBATORS AND LAMINAR FLOW HOODS

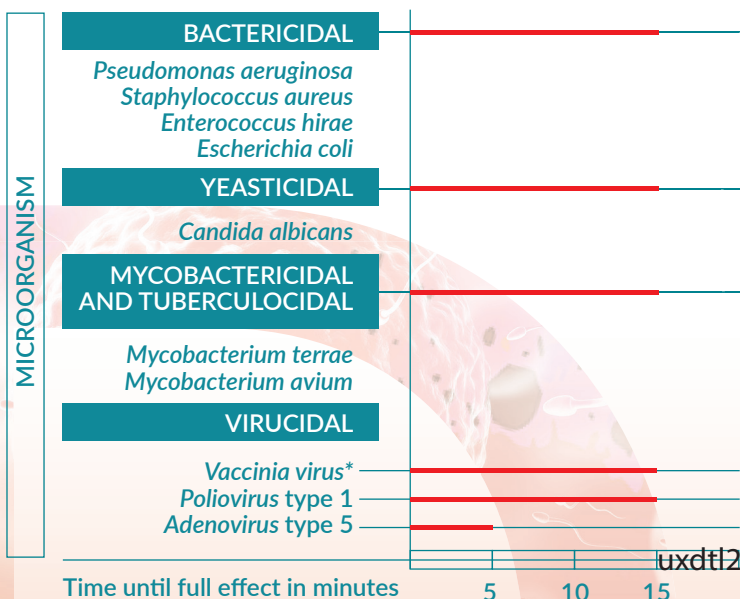

### High quality

- Ready-to-use solution
- Anticorrosive
- Securely sealed
- Valid for 3 years, even after opening
- Long term disinfection/low evaporation rate
- Fast disinfection effect
- Transparent and odorless

### Health and safety

- MEA and HSSA tested
- Alcohol-free
- No VOC release

### Excellent material compatibility

- Stainless steel
- Plastic
- Glass
- Acrylic

### Application:

Follow laboratory's surface cleaning procedure before applying the disinfectant.

**Spray:** Spray and distribute evenly on the surface with a sterile cloth. Wait 15 minutes for the disinfection effect. Wipe the surface to remove any residue.

**Wipes:** Wipe the surface evenly and wait 15 minutes for the disinfection effect. Close the lid after use. Do not reuse wipes.

### Intended use:

Disinfection of CO<sub>2</sub> incubators and laminar flow hoods.



Product reference:  
**OODIH-0050**

Oosafe® Disinfectant for CO<sub>2</sub> Incubators and Laminar Flow Hoods, 50 ml in spray bottle



Product reference:  
**OODIH-1000**

Oosafe® Disinfectant for CO<sub>2</sub> Incubators and Laminar Flow Hoods, 1 Liter bottle with spray



Product reference:  
**OODIH-5000**

Oosafe® Disinfectant for CO<sub>2</sub> Incubators and Laminar Flow Hoods, 5 Liters refill container



Product reference:  
**OODW-70**

Oosafe® Disinfection Wipes, 70 wipes per bucket