Oper cat classic

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Oper cat classic

Catheter protection

Protection of temporary or permanent catheters

Self-adhesive dressing pouch for catheter protection

Oper cat classic is a sterile, disposable, self-adhesive dressing pouch for the **protection of temporary and permanent catheters**.

Oper cat classic is a **non-woven dressing** that enables daily hygiene. It improves patients' asepsis and quality of life.

Oper cat is **easily applied** by healthcare personnel. Patients undergo no discomfort when it is applied and removed.

- 1** Protective sheets to facilitate **rapid and sterile application**, protecting the part of the dressing that covers the catheter outlet.
- 2** **Highly absorbent, non-adhesive** non-woven fabric pad.
- 3** Non-woven fabric, self-adhesive, extensible, highly porous strip ensuring **perfect fixation**.
- 4** **Highly transpirable and extensible non-woven fabric**. The patient's daily hygiene is ensured without having to change the dressing.
- 5** **Transpirable**, highly absorbent pouch, suitable for **maximum asepsis** and cushioning the catheter. The outer layer is water-repellent.

Highly transpirable dressing pouch

Oper cat classic

Catheter protection

Highly transpirable dressing pouch

Presentations

Reference	Measurements	Units per pack	Packs per box	Selling units
0037438	8 x 7.5 cm	25	15	375
0037448	16 x 5 cm	25	14	350

Composition: Upper and lower fixation: adhesive non-woven fabric.
Pad: polypropylene, polyester and viscose mix.

Application for:

Temporary, long-term and permanent catheters, heparin locks, jejunostomy and gastrostomy feeding tubes.

Method of application

1 Remove Oper Cat Classic from individual pack.

2 Insert catheter in pouch.




The open part of the pouch should be towards the patient.

3 Remove the lower silicon-coated protector from the upper non-woven fabric dressing and fix to patient's body.

4 Remove the upper silicon-coated protector from the top of the non-woven fabric dressing and fix completely to patient's body.

5 Once the non-woven fabric dressing is fixed to the patient's body, the lower non-woven fabric dressing is applied:

- If the length of the catheter is similar to the length of the pouch, remove the silicon-coated protector from the non-woven fabric strip and fix to patient.
- If the catheter is shorter, it can be folded to the right size; remove the silicon-coated protector from the strip and fix to patient.

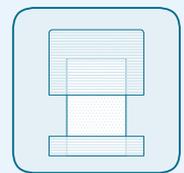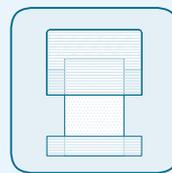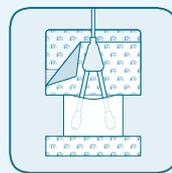Rev.: 2017-03-06

Technical characteristics

Support		Self-adhesive fabric	
Material	100% polyester non-woven fabric	Material	Adhesive non-woven fabric
Grammage	47 ± 5 g / m ²	Adhesive grammage	25 ± 5 g / m ²
Colour	White	Adhesiveness	750 - 1,250 g / 25 mm
		Permeability (MVTR)	≥ 1,400 g · m ⁻² · 24 h ⁻¹ at 37 °C
Upper pad		Lower pad	
Material	Non-woven fabric	Material	Non-woven fabric
Grammage	90 ± 10% g / m ²	Grammage	154 ± 10% g / m ²
Absorption	> 250 %	Absorption	> 710 %

Manufactured and marketed by _____

Iberhospitex S.A. Avda. Catalunya, 4 - 08185 Lliçà de Vall, Barcelona, Spain
Tel +34 93 843 60 34 - Fax +34 93 843 61 11 - info@iht.es - www.iht.es

Information intended exclusively for healthcare professionals