# U 8000 S Ultrafilter

For preparation of clean fluids  
for dialysis

## The U 8000 S Ultrafilter, for Ultrapure dialysis and on-line HDF/HF

- Purification of incoming water for dialysis.
- Production of Ultrapure dialysis fluid
- On-line preparation of sterile substitution fluid\*
- Equipped with the Polyamix™ membrane
- High bacterial and endotoxins retention capabilities, both throughout the treatment and during subsequent sessions.
- Easy to disinfect and rinse\*\*.

\* In combination with an additional single use ultrafilter and with the Gambro AK 100 ULTRA™/AK 200 ULTRA™ machine.

\*\* If the U 8000 S is used in combination with Gambro AK 100 ULTRA™/AK 200 ULTRA™ machine only disinfectants which are recommended in the operator's manual of the AK 100 ULTRA™/AK 200 ULTRA™ should be used. Furthermore the disinfection should follow the procedure described in the operator's manual of the AK 100 ULTRA™/AK 200 ULTRA™.

# U 8000 S Ultrafilter

## Specifications

Components	Material
Membrane	Polyamix™ membrane*
Potting material	Polyurethane (PUR)
Housing and caps	Polycarbonate (PC)
Sterile plugs	Polypropylene (PP)

\* A blend of Polyarylethersulfone, Polyvinylpyrrolidone and Polyamide

Effective membrane area (m²)	2.1
Inner diameter (µm)	215
Wall thickness (µm)	50
Sterilization agent	Steam
Product code	N 50 030

## Performance

Filtration flow	at pressure drop Δp (mm/Hg)	
Q <sub>F</sub> (ml/min)	21°C	37°C
300	43–64	29–43
500	71–107	48–71
700	100–150	67–100

Determined with 0.9% sodium chloride solution.  
Pressure drop Δp is measured between fluid inlet and filtrate outlet.

### Field of application

Bacteria, pyrogen and particle filter for water and dialysis fluid

### Priming volume (ml)

Lumen	152
Filtrate side	306

### Typical retention values for bacteria and endotoxins

	LRV*
Bacterial challenge: Pseudomonas diminuta ATCC 19146 in saline lactose broth. Cell diameter approx. 0.3 µm	>7
Endotoxin challenge: E. coli O55:B5 endotoxin (Whitaker USA)	> 3.5
Natural endotoxin challenge: Endotoxin in water from various places	>3

\*LRV=Logarithmic reduction value  $LRV = \log_{10} \frac{\text{number of organisms in challenge suspension}}{\text{number of organisms in filtrate}}$

### Disinfection\*

<b>Chemical disinfection agent, recommended for the U 8000 S</b>
Peracetic acid (≤0,1%)
Sodium carbonate (≤2%)
Sodium hypochlorite (0,5%)
Citric acid (≤2%)

\*When used with AK 100/200, 200 S and AK 200 ULTRA S, the disinfection routine recommended in Operator's manual of these machines should be followed.

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„Gambro Lundia AB“