Puncture Site Pressure Plaster

Gambro-Plaster® 20 — when patient comfort comes first.

Simple to use. Easy to apply.
Stops bleeding. Relieves pain. Stops blood loss.
Seals the wound. Holds the dressing.


The Gambro Plaster cellulose sponge expands to six times its original volume on contact with blood.

Gambro
Pharmaceuticals
Corporation
Rte. 1, Box 100
Burlington, N.J. 08016


Puncture Site Pressure Plaster

The special cellulose sponge expands rapidly in contact with fluids resulting in immediate pressure on the wound. Provides a rapid and even seal at the puncture site. Can be removed easily once hemostasis has been achieved. Adhesive does not adhere to skin once Gambro Plaster XL has been removed.


Instructions for use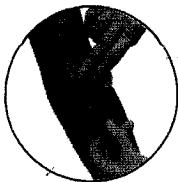

1. Place the cellulose sponge over the puncture site prior to needle removal.
Attach one side of the adhesive tape securely to the skin.
Remove the needle.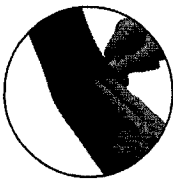

2. Apply pressure to the puncture site and attach the other side of the adhesive tape securely to the skin.

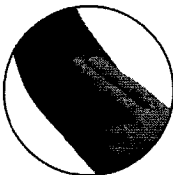3. After one minute the cellulose sponge is fully expanded.
Continue to apply pressure for a period of 15 minutes (or longer if required).

Product specifications

Designation	Size		Contents/ box	Boxes/ container
	Bandage	Sponge		
Gambro Plaster XL	30 x 72 mm	15 x 20 mm	500	16

Sponge: natural cellulosic fibers

Adhesive: crude rubber, polyterpene resin, polybutene

Plastic film: polyvinyl chloride, polyester adipate and zinc stearate polyester sterilized with ethylene oxide

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HCEN7905 © „Gambro Lundia AB“. 1994 m. liepos mėn. Spausdino „Elanders Skogs Grafiska AB“. 3-ioji peržiūra. 2001 m. rugpjūčio mėn.