

FORMALIN FREE FIXATIVE

NEW

SMART WAY FOR DETECTION OF INTESTINAL PARASITES

Paraclin[®] S

Disposable Parasitology System



Paraclin[®] S

Disposable Parasitology System

Formalin free fixative

High clarity of sediment for microscopic examination

Well preserved morphology of cysts and eggs

Quick procedure



Cat number F-099 Tests per kit 150 Shelf life 3 years

Technical specifications

Closed system	The collection tube with a spoon and glass beads to homogenise the sample. The closed system avoids spills, smell and sample contamination.
Double filter	The two filters in the tube: 250 µm and 400 µm produce a clean sediment and allow the optimal recovery of parasites.
Centrifuge filtration	The filtration is performed during the centrifugation, which eliminates the need of adding Ethyl Acetate.

Formalin Free Fixative

Safe Fixative doesn't contain formalin, alcohol and heavy metals. It is classified as non-hazardous.

Small size	The system fits in regular 15 ml size centrifuges buckets. Convenient small tube size reduces the storage space.
-------------------	--

Let's
make
it easy



Procedure

- Collect the faecal specimen
- Vortex or shake the collection tube for 30 sec
- Replace the bottom cap with the sedimentation tube pointing upwards
- Centrifuge for 3 minutes at 700 g
- Discard the collection tube and pour off the supernatant
- Resuspend the sediment with 1-2 drops of water or saline
- Add Lugol's Iodine to enhance the visibility of parasites
- Transfer for examination on a microscope slide

ver: 05/2017



PZ Cormay S.A.
Warsaw Office

Poland, 02-785 Warsaw
303 Pulawska Street

phone: + 48 22 751 79 10
fax: + 48 22 751 79 11

www.cormay.pl
e-mail: office@cormay.pl