Aesculap[®] Neuro-Patch[®] & Lyoplant[®]

Suturable Dura Substitution
Vernähhbarer Duraersatz

Aesculap Neurosurgery

Neuro-Patch[®] & Lyoplant[®]

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Handling characteristics

- high elasticity and flexibility
- can be cut to any desired shape and size
- very easy to suture
- no suture pull-out

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Product features

- liquid-tightness prevents CSF leakages
- microporous structure hence rapid host tissue population
- biocompatibility
- high tensile strength

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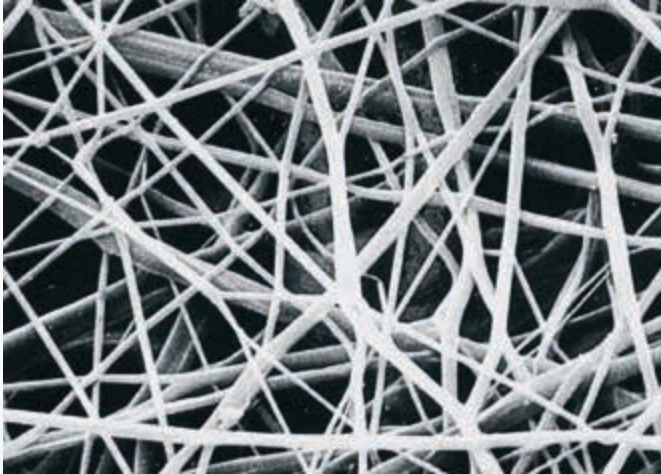
Elastic. Well-tolerated. Non-absorbable.

Neuro-Patch[®] is a type of fine fibred microporous fleece manufactured from a highly purified polyesterurethane.

An electronically controlled procedure has been developed for producing Neuro-Patch[®], in which a dissolved polyesterurethane polymer is sprayed from specialised jets. This produces fibres which are captured at defined angles, producing a fleece-like structure.

The fine fibred microstructure supports the rapid infiltration of connective tissue cells.

Product Sample Card C87802
including a product sample.

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Biological, suturable dura substitution
Biologischer, vernähbarer Duraersatz

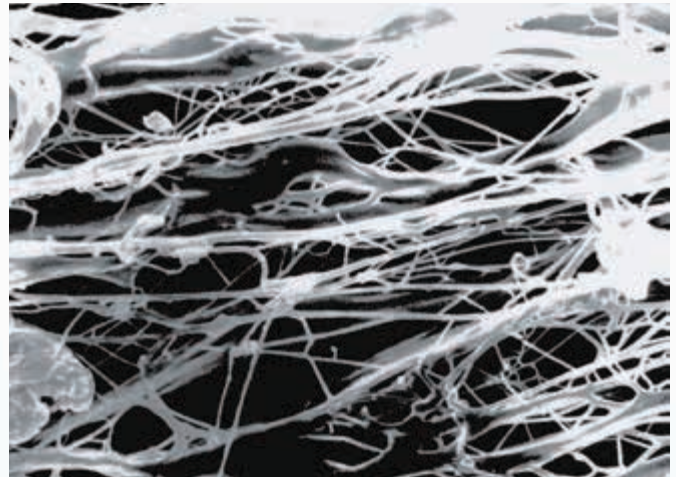
Functional. Tearproof. Absorbable.

Lyoplant[®] is a pure collagen implant that is produced from bovine pericardium. This highly purified collagen is known for its low propensity to cause immunological reactions.

The special and comprehensive manufacturing process eliminates all non-collagenous substances such as fats, enzymes and other non-collagenous proteins.

A controlled lyophilisation (freeze drying) process supports that the loose fibre structure of Lyoplant[®] is preserved to offer appropriate conditions for integration after implantation.

After implantation, Lyoplant slowly degrades enzymatically and is substituted by body's own connective tissue.

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Product Sample Card C87902
including a product sample.

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* To improve smoothness and flexibility, Lyoplant® material is rehydrated prior to implantation.

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Mode of application

To achieve tension-free embedding, Neuro-Patch® and Lyoplant® should be cut as closely as possible to the defect size.

Neuro-Patch® and Lyoplant® should be fixed with non-absorbable suture material (polyester, polypropylene). Additionally, it may be secured with fibrin glue.

Atraumatic round-bodied needles allow suturing without significant damage to the implant.

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