Kamfenol / Kamfenolis

CE 0197

Instructions for use

Product destination:

THE PRODUCT IS FOR PROFESSIONAL USE ONLY.

Kamfenol is used as a temporary dressing in root canal treatment and periapical alterations in dead teeth.

Directions for use:

Apply the liquid on walls of the pulp chamber using wadding tampon. Close the cavity through the temporary restoration for 2-5 days. Thereafter remove the dressing and restore the tooth.

Contraindications:

Do not use in patients with hypersensitivity on some of the ingredients. Do not use in pregnant women. Do not use after expiry date.

Precautions:

Do not use in the tooth alive. Do not apply for more than 5 days. Do not use in case of treatment with antibiotics, chemotherapy, corticosteroids as well as hydroxy and iodine iontophoresis.

Use cofferdam while handling the medical device. Irritating to eyes and skin. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of water. If skin irritation occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Do not use the device if any signs of damage are observed. Expiry date does not change after opening under condition of tightly closing after each use.

Storage conditions:

Store in a dry place, in original and tightly closed packaging, at temperature up to 25°C. Protect from moisture.

Expiry date:

Expiry date for use is printed on the immediate packaging.

Handling and storage of packaging:

Empty package recycle according to applicable provisions.

Composition:

Camphor, phenol, ethanol (96%).

Immediate packaging:

Glass bottle with dropper containing 5 ml, 10 ml or 20 ml of the product.

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PFO „VETOS-FARMA“ Ltd.