




# Pocket Vibrator

## INSTRUCTION MANUAL

**Nr. 8** Specially designed for deaf and hearing impaired people.

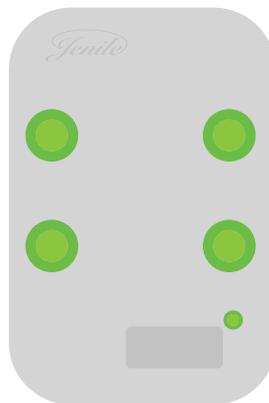



All your products are already connected to each other.


# Summary :

- 4** : The contents of your package
- 5-10** : Pocket vibrator VP15

# The contents of your package



(A)

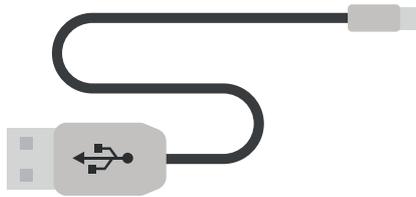

(B)




(C)

**Nr. 7**

- (A) - Vibrator Pocket
- (B) - USB connection port
- (C) - Manual instruction in english

# Pocket vibrator VP15

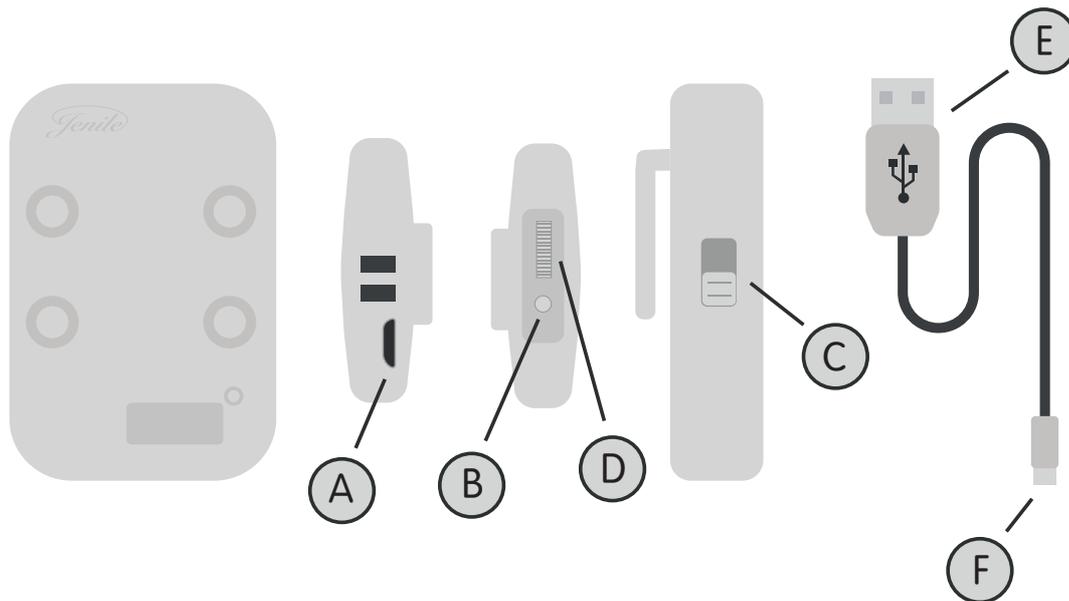

(A) - Micro USB charger port

(B) - connection plug

(C) - On / Off

(D) - Dimmer of vibration

(E) - USB connection port

(F) - Micro USB connection port

## Technical Specifications

### **Nr.2** Wireless device

Rechargeable battery

Vibrator's weight: 60g

### **Nr.5** Distance between the transmitters and the vibrator: 200m in free fields

Quantity of connected products: 20 units max

### **Nr.7** Power supply: 3.7V 2200mAh Lithium battery.

Charge voltage must be limited to 4.2V

Standby current: <4,5mA

Operating current: <800mA

Radio frequency: 433Mhz

Decoding format: MCU


### **Nr.6** Vibrator's dimensions (height x width x depth): 6x8x1,3cm

Autonomy: 1 month (depending of the use)


2 years guarantee (battery 1 year)

## Use


**Nr.2** Thanks to your Pocket vibrator VP15, you receive the signal from the Jenile doorbell and also all the other calls, safety signals and technology Jenile. 4 rhythm of vibration and 4 colors indicate the call sources or the alerts.

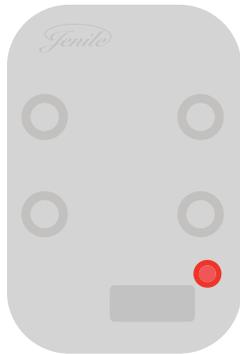

Red LED : low battery,  
you need to recharge your vibrator

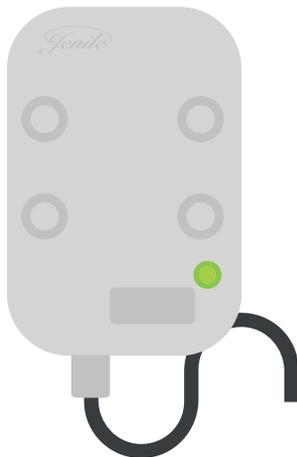

Green LED : battery is in charge

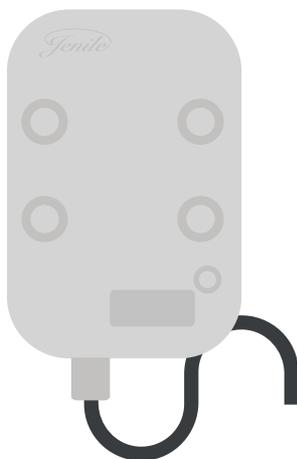

No LED lightened up :  
the charge is complete


Warning : it is also possible that no LED is lightened up because the battery is completely empty. Therefore, you need to charge it.

## Test

The Jenile Pocket vibrator VP15 is easy to test. If the Pocket vibrator does not work in accordance with the instructions below, we invite you to contact your seller. To test your Pocket vibrator press on your Jenile doorbell.




If the Vibrator light up, but no vibration please check (D) maybe it put on minimum.

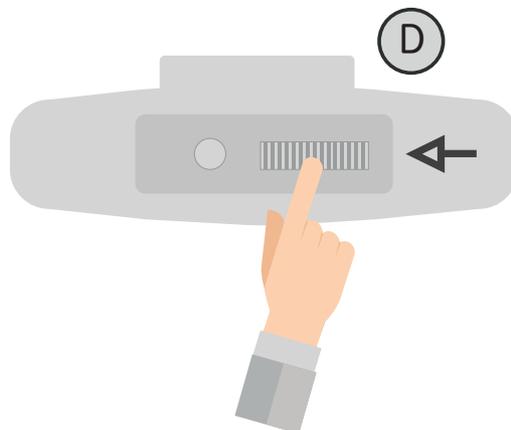


If the green LED at the bottom right of your device does not light up, it means that your device is fully charged.

# Settings


if you want to complete this pack with other Jenile transmitters (see products on page 16), then make adjustments to connect your devices between each other. To connect all your products, please follow the instructions carefully.

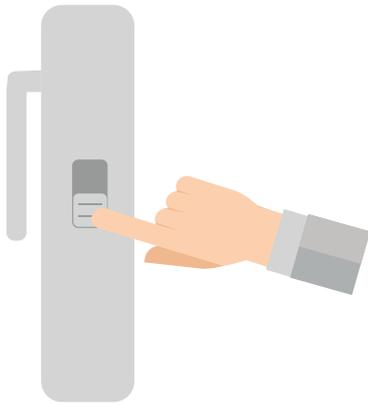

1. Activate the vibrator by pushing down the ON button.

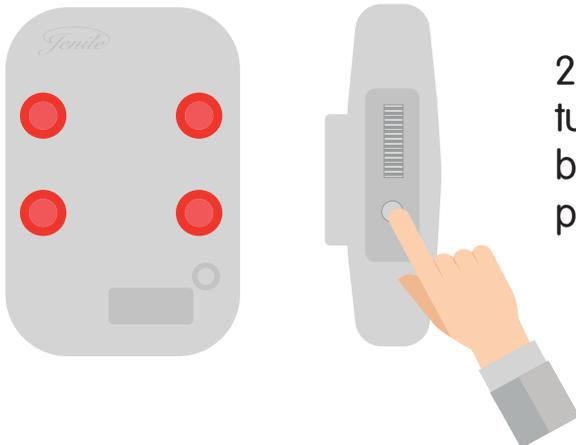


2. Press the plug button (C) until the vibrator turns red (WARNING: do not hold down the button too long, you are at risk of deleting all previous connections)

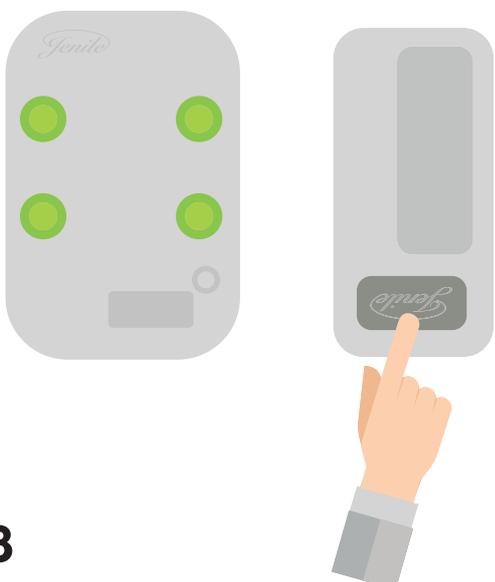


3. Immediately actuate the transmitter after having pressed the plug (ring the bell, press on the remote control, shout in the baby crying detector, ...)


### 3 possibilities:

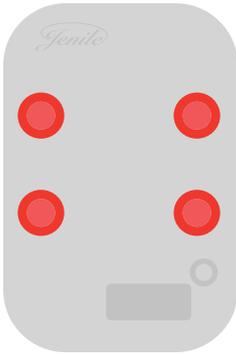

Vibrator turns red but doesn't vibrate.  
The connection has ... it hasn't received  
the transmitter's signal

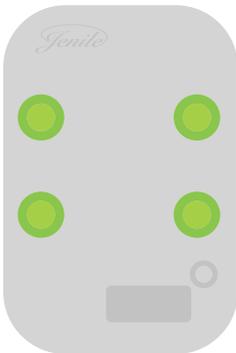


The vibrator flashes in a color (green, blue, red  
or white, depending on the alert) and vibrates.  
The connection is successful

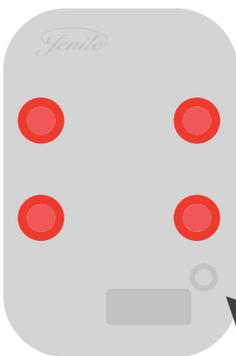

3 times

The red light flashes 3 times, it doesn't vibrate.  
The connection has failed because you  
pressed too long on the plus button ©.  
You erased the memory and have to restart  
all the connections.

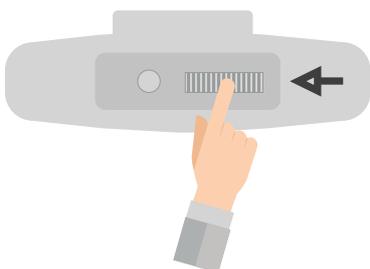


4. Choose the vibration level that suits you best  
with the variator on the top of your vibrator

# Signals and colors


The Pocket vibrator VP15 can be used with all the transmitters from the Jenile range.

For all alerts, the Pocket vibrate and flash at the same time. The color at the of the light as well as the vibration rhythm will indicate the alert type. For doorbell it's in green, 2 vibration with 5 repetitions (more colors page...).

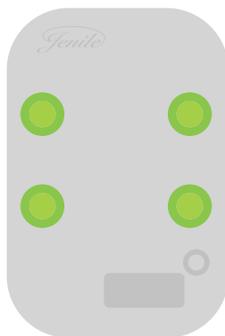


## **GREEN and medium vibrations (x2 with 5 repetitions) : Bells**

- Door bells
- Video bells
- Video Gate
- Acoustic transmitter
- Call button, ...

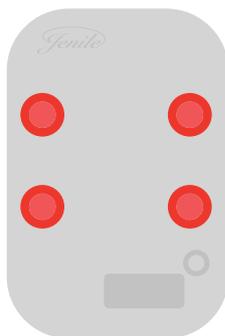


## **RED and quick vibrations (x3 with 4 repetitions) Emergencies**

- Smoke detector
- Gas detector,
- water flood detector, ..

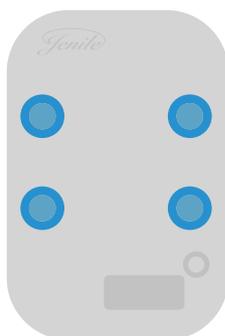

## **BLUE and slow vibrations (x1 with 5 repetitions)**

- Calls
- Text messages
- Phone, fax, ...

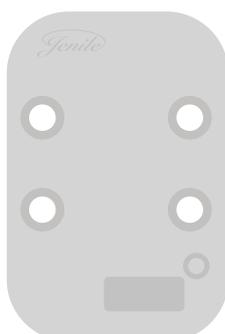

## **WHITE and very quick vibrations (x4 with 4 repetitions)**

- Door/window opening detector
- Motion detector
- Computer calls (skype, oovoo, ..)

*Jenile*  
Cr   par et pour les sourds

[www.jenile.com](http://www.jenile.com)



CE