



06530290001V6.0

Cell Wash Solution II / Acid Wash **cobas**[®]

CellCln2/Detergent 2

Order information

REF	CONTENT	Analyzer(s) on which kit(s) can be used
04880307 190	Cell Wash Solution II / Acid Wash (2 x 1.8 L)	Roche/Hitachi MODULAR Roche/Hitachi cobas c

English

Intended use

Cell Wash Solution II / Acid Wash is used as acid wash solution for reaction cells on Roche/Hitachi systems.

Summary

Routine reaction cell (cuvette) washes are required due to potential interference from other reagent/sample mixtures. This procedure maintain the integrity of patient sample results.

Reagents - working solutions

Citric acid monohydrate: 310 mmol/L; buffer; detergent

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Danger

H314 Causes severe skin burns and eye damage.

Prevention:

P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.

Response:

P301 + P330 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
+ P331

P303 + P361 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.
+ P353

P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing.
+ P310 Immediately call a POISON CENTER/ doctor.

P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.
+ P338 Continue rinsing. Immediately call a POISON CENTER/ doctor.
+ P310

Disposal:

P501 Dispose of contents/container to an approved waste disposal plant.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Reagent handling

Ready for use

Storage and stability

Shelf life at 15-25 °C: See expiration date on labels.

On-board stability: 12 weeks

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Roche/Hitachi analyzers and reagents.

See appropriate package inserts and operator's manuals for additional required materials.

Assay

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets and method sheets of all necessary components.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see <https://usdiagnostics.roche.com> for definition of symbols used):

DETERGENT 2	Detergent 2
CONTENT	Contents of kit
REAGENT	Reagent
CALIBRATOR	Calibrator
→	Volume after reconstitution or mixing
GTIN	Global Trade Item Number

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2017, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

