

ECO-D

EcoTergent

cobas®

Order information

REF	CONTENT	Analyzer(s) on which kit(s) can be used
06544410 190	EcoTergent (12 x 59 mL)	Roche/Hitachi cobas c 501/502/ Self-Contained ISE Support Box
08063168 001	Bottle Set for cobas 8000 ISE (6 x 590 mL bottles, empty)	Self-Contained ISE Support Box

English

Intended use

EcoTergent is an additive to the reaction bath to reduce surface tension on cobas c systems.

Summary

EcoTergent is added to the reaction bath. It acts as a surfactant to minimize the formation of bubbles that could potentially interfere with the photometer readings.

Reagents - working solutions

Detergent

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Warning

- | | |
|------|---|
| H315 | Causes skin irritation. |
| H319 | Causes serious eye irritation. |
| H410 | Very toxic to aquatic life with long lasting effects. |

Prevention:

- | | |
|------|--|
| P264 | Wash skin thoroughly after handling. |
| P273 | Avoid release to the environment. |
| P280 | Wear protective gloves/ eye protection/ face protection. |

Response:

- | | |
|-------------|---|
| P337 + P313 | If eye irritation persists: Get medical advice/attention. |
|-------------|---|

- | | |
|------|-------------------|
| P391 | Collect spillage. |
|------|-------------------|

Disposal:

- | | |
|------|--|
| P501 | Dispose of contents/container to an approved waste disposal plant. |
|------|--|

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Reagent handling

Ready for use

ACTION REQUIRED for use on Self-Contained ISE Support Box:
Use 3 of the ECO-D bottles to manually fill one of the empty bottles of the Bottle Set to approximately 180 mL.

Storage and stability

EcoTergent

Store protected from light.

Shelf life at 15-25 °C: See expiration date on reagent.

On-board in use on the analyzers cobas c 501/502/Self-Contained ISE Support Box: 12 days

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Assay

Use EcoTergent as specified in the respective instructions for use of the system reagents.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see <https://usdiagnostics.roche.com> for definition of symbols used):

CONTENT

Contents of kit



Volume after reconstitution or mixing

GTIN

Global Trade Item Number



FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2017, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

