

Internal Standard Insert - ISE

cobas®

 • Indicates **cobas c** systems on which reagents can be used

Order information

Internal Standard Insert - ISE (Set of 20)

Cat. No. **05979854** 190Roche/Hitachi **cobas c** systems

cobas c 311	cobas c 501
--------------------	--------------------

English
Intended use

The Internal Standard Insert - ISE is intended for use with the Internal Standard Gen.2 bottle. The insert prevents evaporation of Internal Standard. Its use is recommended in case of proven or potential Internal Standard evaporation.

Summary

The Internal Standard Insert is a plastic tube to be inserted into the Internal Standard Gen.2 bottle.

Precautions and warnings

For in vitro diagnostic use.

Disposal of all waste material should be in accordance with local guidelines.

Storage and stability

There is no expiration for the Internal Standard Insert. Do not store the Internal Standard Insert in direct sunlight. Store at room temperature.

Materials provided

See "Order information" section.

Materials required (but not provided)

ISE Internal Standard Gen.2 (5 x 600 mL), Cat. No. 04522320 190

Handling

Follow the steps described below for insert use.

Step 1

Use a **new** insert for each new bottle. Place the insert as shown in the picture, and push it slowly downwards into the bottle.

Note To avoid overflow of the Internal Standard, do not cover the hole of the insert as it is being pushed down.


Step 2


Make sure the insert is pressed into the bottle as far as possible.


Step 3.1 (for **cobas c 501 analyzer)**

Place the bottle with insert into its slot. Push the bottle to the right as far as possible and fully to the stop.



Incorrect placement

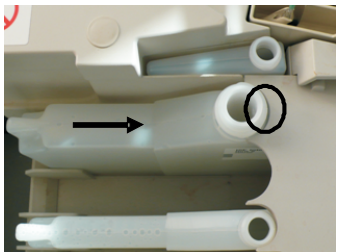


Correct placement

Step 3.2 (for **cobas c 311 analyzer)**

Step 4

Once the bottle is on the analyzer, confirm that the insert is correctly in place.

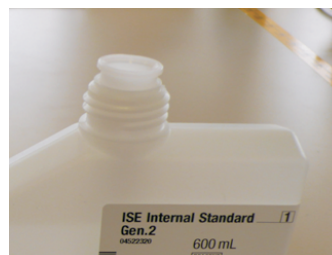

Incorrect placement



Correct placement

Step 5

If you intend to use a 2nd bottle on **cobas c** 501 systems, follow the same steps for insert use.

Note Use a new Internal Standard Insert for each Internal Standard Gen.2 bottle.

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

Significant additions or changes are indicated by a change bar in the margin.

© 2009, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:

Roche Diagnostics, Indianapolis, IN

US Customer Technical Support 1-800-428-2336

