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Scaling tips

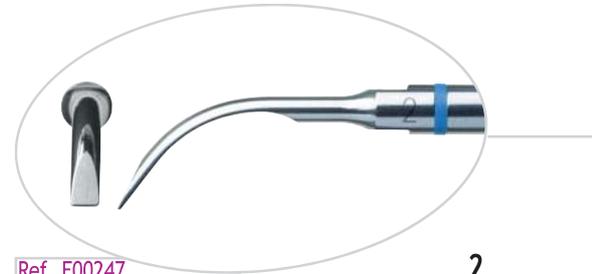
These four supra- and sub-gingival scaling tips are ideal for treating commonly encountered cases (pockets <3-4mm). All tips come with the new Spray Control System (Satelec exclusive).

The tip must be oriented tangentially to the surface being treated and must be moved in a back-and-forth sweeping motion, over the whole surface, without excessive lateral pressure. Removing thick deposits may require using the Boost mode for short periods. (19)


Ref. F00246

1

Ref. F00247

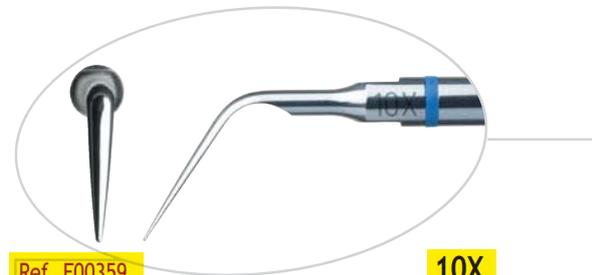2

3.2.

Ref. F00359

10X


3.1.

Ref. F00254

10Z

POWER SETTINGS

	P-Max Newtron Prophy Max Newtron	P5 Newtron	SP Newtron	Suprasson P5 Booster
No. 1 tip	8 to 10	14 to 15	7 to 8	7 to 9
No. 2 tip	8 to 10	14 to 15	7 to 8	7 to 10
No. 10X tip	6 to 8	12 to 14	6 to 7	7 to 8
No. 10Z tip	6 to 8	12 to 14	6 to 7	7 to 8

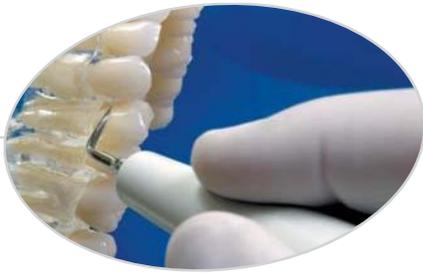No. 1 tip: "Universal". Recommended for treating simple cases and gross supra-gingival scaling. Tangential orientation with respect to the tooth surface. To-and-fro sweeping motion to "push" and "pull" calculus without damaging the enamel.

No. 2 tip: "Voluminous calculus". Recommended for removing voluminous supra-gingival deposits. Apply flat end to surface of teeth.

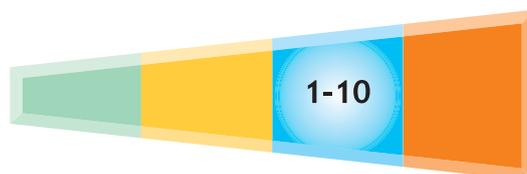
3.2.

No. 10X tip: "Interproximal". Recommended for the treatment of interproximal spaces and for supra-gingival scaling. Anatomic shape allows quick, efficient hand movements.

3.1.

No. 10Z tip: "Sub-gingival". New design for the 10Z for sub-gingival scaling, this instrument is recommended for scaling medium pockets (<4mm). The new version is more powerful as its amplitude has been increased by 75% and the working edges are longer.

Color Coding System - Newtron Technology
Identification of power modes by color coding.

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Tips

Mastering ultrasound does not stop at devising generators and control software, it also means designing the appropriate accessories that will give them optimum vibration power and guarantee their reliability.

Our instruments are made from the best alloys, exclusive to **Satelec**, using patented procedures.

We offer you a range of different coatings (diamond, thermal treatments) or materials (carbon) so you can choose the most suitable and efficient instrument for each material, giving you better performance.

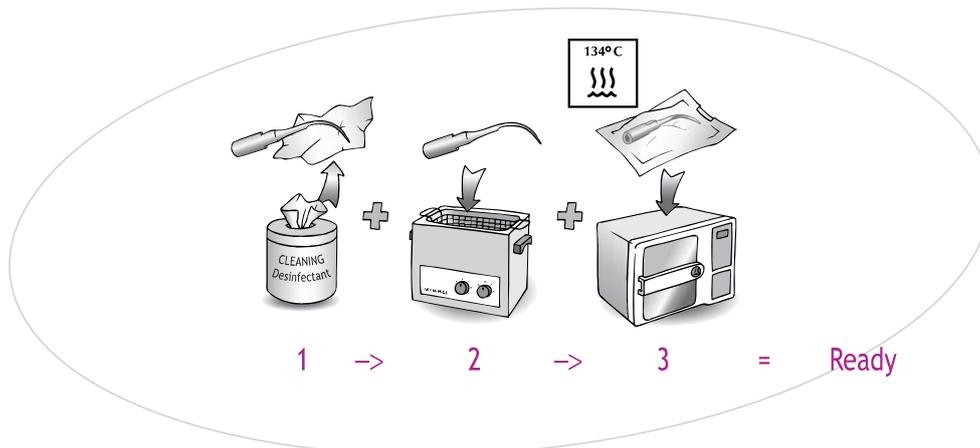
Satelec's expertise in tuning all its ultrasonic instruments is second to none. Our transducers are probably the best in the world and certainly the only ultrasonic motors capable of driving all these tips at maximum performance, whatever their type or composition.

Advantages

- **Corrosion resistant:** the steel alloy we have chosen does not corrode and is resistant to decontamination products as well as to irrigation solutions.
- **Tensile strength:** the manufacturing procedures are designed, and the tensile properties adjusted, to suit the treatments for which they are intended.
- **Sterilization:** the tips are suitable for autoclave sterilization, to comply with ISO-11134, as are the handpieces and accessories (wrenches, storage kits, etc.):

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
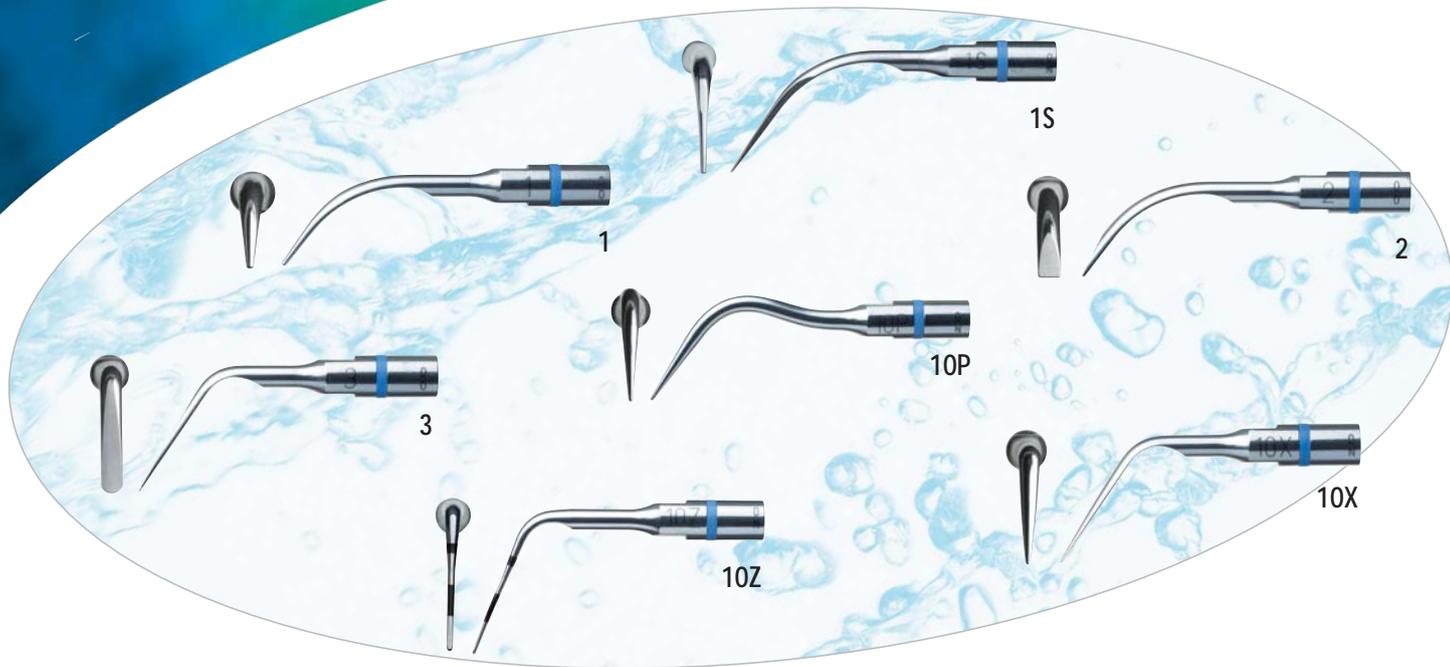
							
	n°1	n°1S	n°2	n°3	n°10P	n°10X	n°10Z
	Ref. F00246	Ref. F00245	Ref. F00247	Ref. F00248	Ref. F00253	Ref. F00359	Ref. F00254
P-Max Newtron XS	14-15	14-15	14-15	14-15	14-15	12-14	12-14
P5 Newtron / P5 Newtron XS	14-15	14-15	14-15	14-15	14-15	12-14	12-14
P-Max Newtron/Lux Prophy Max Newtron/Lux	8-10	8-10	8-10	8-10	8-10	6-8	6-8
P5 Booster	7-9	7-10	7-10	7-8	7-8	7-8	7-8
Module SP Newtron (LED)	7-8	7-8	7-8	7-8	7-8	6-7	6-7
Suprasson P-Max*/Lux* Prophy Max*/Lux*	5-9	5-10	5-10	5-8	5-8	1-5	1-4
Module SP4055*/Lux*	5-9	5-10	5-10	5-8	5-8	1-5	1-5
Module SP3055*/B*	5-9	5-10	5-10	5-8	5-8	1-5	1-4

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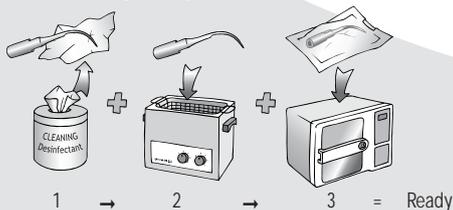The **no.10Z** tip is now marked with a gauge based on the principle of the Dutch Periodontal Screening Index (University of ACTA, Amsterdam/Belgian Association of Periodontology) to determine quickly the patient's state of health during prophylactic and initial treatments, or periodontal maintenance.