## IFU FOR FILL PORT CLEANER

Company name: JS Medicina Electrónica SRL

Common name: DIESTRO FILL PORT CLEANER

- **Intended Use**

The Fill Port Cleaner is a product designed to absorb the remains of biological substance that are deposited in the outer part of the capillary sampling. Furthermore, to protect the user from contact with said biological substance during normal operation of the analyzer.

The life of The Fill Port Cleaner is calculated for 800 samples for an estimated consumption of 100 samples day or 3 months for a consumption of 10 samples day.

- **Reference Number: IN 0050**

- **Procedure**

1. When the analyzer is off, open the front panel.
2. Carefully lift the Retractable fill port to take a horizontal position, remove The Fill Port Cleaner used and discarded according to applicable regulations related to potentially infectious biological waste.
3. Clean the needle with a disposable cloth or towel moistened with Diestro intensive wash solution.
4. Remove the security seal of the new cleaning module.
5. Present the new Diestro Fill Port Cleaner in front of the guide position, make sure it is in the correct position and gently slide it on till hit the central wall of the analyzer.
6. Enter the retractable fill port into the analyzer.
7. Place the front of the analyzer in its position and continue using the analyzer normally.


- **Caution and Disposal**

The needle and the used Diestro Fill Port Cleaner may contain potentially infectious waste: be cautious, wear gloves and avoid splashing.

Discard the cloth or towel and the used Diestro Fill Port Cleaner in accordance with the regulations applicable at your laboratory.

## • Storage & Handling

Store the fill port cleaner at 5°C -35°C at room temperature.  
**Wear gloves at any time.**

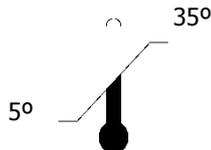Obelis s.a  
53 Boulevard Général Wahis  
1030 Brussels, BELGIUM  
Tel: +(32) 2. 732.59.54  
Fax: +(32) 2.732.60.03  
E-Mail : mail@obelis.net

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