

IFU FOR SPARE TUBING KIT

Company name: JS Medicina Electrónica SRL

Common name: DIESTRO

Reference Numbers: RE0319; RE0320; RE0321; RE0322; RE0323;
RE0324; RE0325; RE0326; RE0327; RE0328; RE0329; RE0330;
RE0331

- **Intended Use**

The Spare tubing kit are designed and intended primary for use on Electrolyte analyzer brand DIESTRO to bring the following solutions and samples respectively to different internal points like reagents, human Blood, Serum, Plasma and Urine.

- **Storage & Handling**

Store at 5°C -35°C. Do not freeze.

- **Procedure**


1. Disconnect the tube of the pack.
2. Empty all internal tubes of the analyzer
3. Disconnect the power supply of the analyzer from the voltage line.
4. Open the front panel of the analyzer.
5. Disconnect all internal tubes and replace them with new ones.
6. Connect the tubes of the pack to the analyzer.
7. Switch on the equipment and performs a purge.
8. Perform a Two points calibration and verify that the values of slopes are correct.



* Perform this operation using gloves and eyeglasses

- **Disposal**

Dispose of according to the safety and environmental regulations applicable at your laboratory.

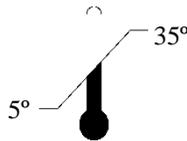

Obelis s.a
53 Boulevard Général Wahis
1030 Brussels, BELGIUM
Tel: +(32) 2. 732.59.54
Fax: +(32) 2.732.60.03
E-Mail : mail@obelis.net



Fabricado por / Manufactured by
JS Medicina Electrónica S.R.L.
Bolivia 462 (B1603CFJ) - Villa Martelli - Provincia de Buenos Aires - República Argentina
Tel/Fax.: (54 11) 4709-7707 - Email: info@jsweb.com.ar - Web: www.jsweb.com.ar
Dirección Técnica: Farm. Marcelo Miranda - Matrícula Nacional N°13104 - Matrícula Provincial N°15964
Industria Argentina / Made in Argentina

Fecha de última revisión / Last revision: 01/2016.-