TruCal CRP

Calibrator set for use in tests for quantitative in vitro determination of C reactive protein (CRP) on photometric systems

Order Information

1 7000 99 10 039 5 x 2 mL

Description

TruCal CRP is a set of five liquid stable calibrators with different levels. The calibrators are based on human blood material (serum).

Storage

The calibrators both opened and unopened must be stored at 2 – 8°C.

Stability

Unopened: Until the end of the indicated month of expiry

Opened: At least 3 months

Proper storage and handling of this product must be observed.

Warnings and Precautions

- Each individual blood donation used for production of TruCal CRP was found to be non-reactive when tested with approved methods for HBsAg, anti-HIV 1+2 and anti-HCV. As there is no possibility to exclude definitely that products derived from human blood transmit infectious agents, it is recommended to handle the calibrator with the same precautions used for patient specimens.
- Contains sodium azide (0,95 g/L) as preservative, Do not swallow! Avoid contact with skin and mucous membranes.
- Please refer to the safety data sheets and take the necessary precautions for the use of calibrators and controls.
- For professional use only!

Preparation

TruCal CRP calibrators are liquid and ready to use.

Calibrator Values


The assigned values of TruCal CRP have been made traceable to the ERM®-DA474/IFCC reference material. Calibrator values listed below are specific for this lot number of calibrator only.

Literature

- Stenman UH, Standardization of immunoassays, In: Price CP, Newman DJ, editors, Principles and practice of immunoassay, New York: Stockton Press; 1997, p. 243-68,
- Dati F, Reference materials and guidelines for standardization of methods in laboratory medicine, In: Thomas L, editor, Clinical laboratory diagnostics, 1st ed, Frankfurt: TH-Books Verlagsgesellschaft; 1998, p. 1393–1401,
- Biosafety in Microbiological and Biomedical Laboratories, U,S, Department of Health and Human Services, Washington 1993 (HHS Publication No. [CDC] 93–8395).

Waste management

Please refer to local legal requirements.

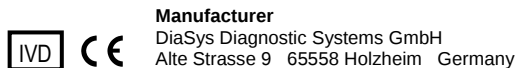	Lot No.	Expiry Date	Calibrator Value
TruCal CRP Level 1	26869	2020-09	0.950 mg/dL
			9.50 mg/L
TruCal CRP Level 2	26870	2020-09	1.98 mg/dL
			19.8 mg/L
TruCal CRP Level 3	26871	2020-09	5.43 mg/dL
			54.3 mg/L
TruCal CRP Level 4	26872	2020-09	14.7 mg/dL
			147 mg/L
TruCal CRP Level 5	26873	2020-09	28.9 mg/dL
			289 mg/L

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Literatur

- Stenman UH, Standardization of immunoassays, In: Price CP, Newman DJ, editors, Principles and practice of immunoassay, New York: Stockton Press; 1997, p. 243-68.
- Dati F, Reference materials and guidelines for standardization of methods in laboratory medicine, In: Thomas L, editor, Clinical laboratory diagnostics, 1st ed, Frankfurt: TH-Books Verlagsgesellschaft; 1998, p. 1393–1401.
- Biosafety in Microbiological and Biomedical Laboratories, U,S, Department of Health and Human Services, Washington 1993 (HHS Publication No. [CDC] 93–8395).

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