



03588416001 V6.0

Multiclean / Sample Cleaner 1

cobas®

Order information

| REF | CONTENT | Analyzer(s) on which kit(s) can be used |
|--------------|---|---|
| 04880277 190 | Multiclean (2 x 2 L) | Roche/Hitachi 902 analyzer Roche/Hitachi cobas c analyzer |
| 04708725 190 | Sample Cleaner 1 (SmpCln1) (12 x 59 mL) | Roche/Hitachi cobas c 311/501 analyzers |
| 05352991 190 | Sample Cleaner 1 (SmpCln1) (12 x 20 mL) | Roche/Hitachi cobas c 502/701/702 analyzers |

Some analyzers and kits shown may not be available in all countries. For additional system applications, contact your local Roche Diagnostics representative.

English

Intended use

Wash solution for sample probes or reaction cells on Roche/Hitachi systems.

Summary

Sample probe and reaction cell washes may be required due to potential interference from other samples. These special washes maintain sample integrity.

Reagents - working solutions

Sodium hydroxide solution 1 mol/L

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the European directive 1999/45/EC:



C Corrosive (sodium hydroxide)

R 34 Causes burns.

S 26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

S 36/37/39 Wear suitable protective clothing, gloves and eye/face protection.

S 45 In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

Contact phone: all countries: +49-621-7590, USA: +1-800-428-2336

Reagent handling

Ready for use

Note

Over time, a slight discoloration or small particles may be observed on the bottom of the bottle. The discoloration or particles do not affect reagent performance.

Storage and stability

Shelf life at 15-25 °C: See expiration date on labels

04708725 190, 05352991 190

On-board in use and refrigerated on the analyzer: 4 weeks

04880277 190

After opening: 12 weeks
Store Multiclean capped when not in use.

On-board in use and refrigerated on the analyzer: 4 weeks

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Roche/Hitachi analyzers and assay reagents.

See appropriate Method Sheet and operator's manuals for additional required materials.

Assay

Use Multiclean / Sample Cleaner 1 as specified in the respective instructions for use of the system reagents.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets and method sheets of all necessary components.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard.

CONTENT

Contents of kit

REAGENT

Reagent

CALIBRATOR

Calibrator



Volume after reconstitution or mixing

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Significant additions or changes are indicated by a change bar in the margin.

© 2013, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

