

CleanCart® cartridge

Safe and easy maintenance and disinfection of the dialysis fluid path

Gambro's response to the need of a user- and environment-friendly but still effective disinfection, decalcification and cleaning procedure is the effective CleanCart cartridge. The CleanCart concept prevents safety hazards.

Regular dialysis maintenance and machine disinfection assure patients' well-being and the best treatment delivery requiring less machine operation from clinic staff. The CleanCart cartridge makes it easy to safely combine heat disinfection with decalcification or cleaning.

After use, the CleanCart cartridge can safely be discarded without posing any threat to the environment.



CleanCart-A koloneleje yra 13 gr. sauso natro karbonato miltelių. Kartu su karstos dezinfekcijos programa skirta dializės aparato skysčių dalies valymui

CleanCart® –A

The CleanCart –A cartridge contains 13 grams of anhydrous sodium carbonate powder (in compliance with Ph.Eur., USP). It is used in combination with the heat disinfection program to clean the dialysis machine fluid path. The sodium carbonate solution removes organic deposits, fats and proteins while circulated in the fluid path.

CleanCart® –C

The CleanCart-C cartridge contains 32 grams of citric acid anhydride powder (in compliance with Ph.Eur., USP). The CleanCart –C cartridge is used in combination with the heat disinfection program to both decalcify and disinfect the dialysis machine fluid path simultaneously. The citric acid solution obtained removes precipitated salts of calcium and magnesium while circulated throughout the fluid path.

CleanCart-C koloneleje yra 32 gr. suamos citrinos rūgšties miltelių. Kartu su karstos dezinfekcijos programa, skirta dializės aparato skysčių dalies dekalifikavimui.

CleanCart-A

CleanCart-C

Action (in combination with the heat disinfection program)	Disinfection & cleaning of the fluid path from organic deposits, fats, and proteins	Disinfection & decalcification of the fluid path
Cartridge content (PH. Eur., USP)	Anhydrous sodium carbonate powder 13 grams	Anhydrous citric acid powder 32 grams
Total cartridge weight	Approx. 37 grams	Approx. 56 grams
Cartridge and label material	Recyclable polypropylene (PP)	Recyclable polypropylene (PP)
Recommended usage	Refer to operator's manual for respective dialysis machine	Refer to operator's manual for respective dialysis machine
PH (prepared solution)	Approx. 11	Approx. 2
Residual test needed	No	No
Biodegradable content	Yes	Yes
Storage conditions	Store below +30°C	Store below +30°C
Shelf life	24 months from date of manufacture	24 months from date of manufacture
Product code	101769	101770
Packaging	Box of 30 cartridges	Box of 30 cartridges
Box dimensions	255 X 245 x 80 mm	255 X 245 x 80 mm
Total box weight	Approx. 1100 grams	Approx. 1700 grams